# MOUNTING- INSTRUCTION

**EN ISO 9001 EN ISO 13485**

SystemCarrier compact Art. No. 20180-100, SystemCarrier compact plus Art. No. 20180-101

All rights to this Mounting Instruction, in particular rights of duplication, dissemination and translation, are reserved. No part of this Mounting Instruction may be reproduced in any form (by photocopying, microfilming or other methods) or processed, duplicated or disseminated by the use of electronic systems without the written consent of Erbe Elektromedizin GmbH.

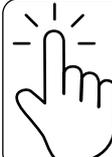
Product availability and regulatory clearances differ by region. Certain Units/Modules described in this document may not be cleared by the U.S. Food and Drug Administration (FDA). Availability and regulatory status vary by country.


The information contained in this Mounting Instruction can be changed or expanded without prior notice and without obligation on the part of Erbe Elektromedizin GmbH.

The illustrations in this Mounting Instruction may differ in some details from your product.

Copyright © Erbe Elektromedizin GmbH, Tübingen 2025

# What would you like to do?

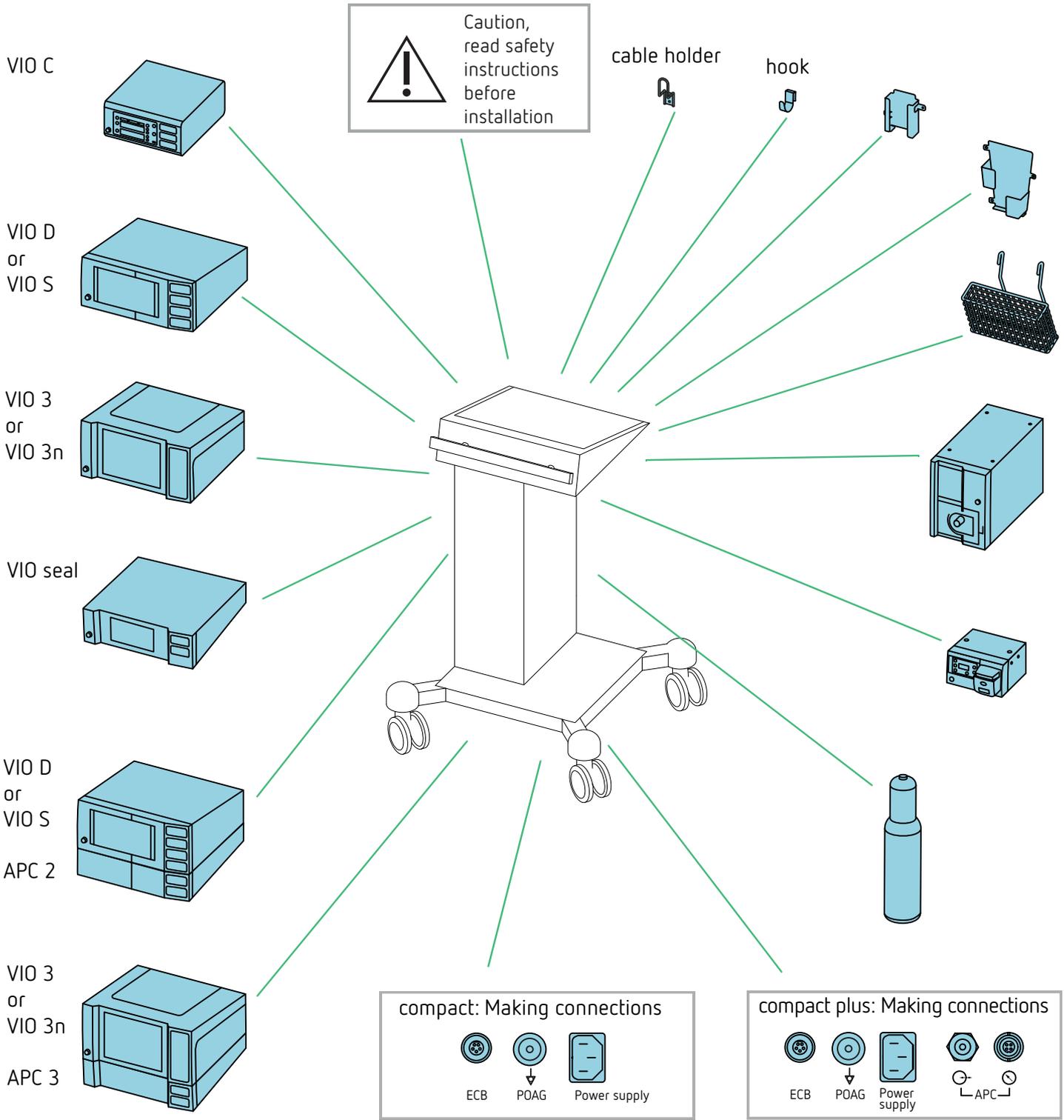 For eFU : Click on the accessory / unit to be mounted.

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
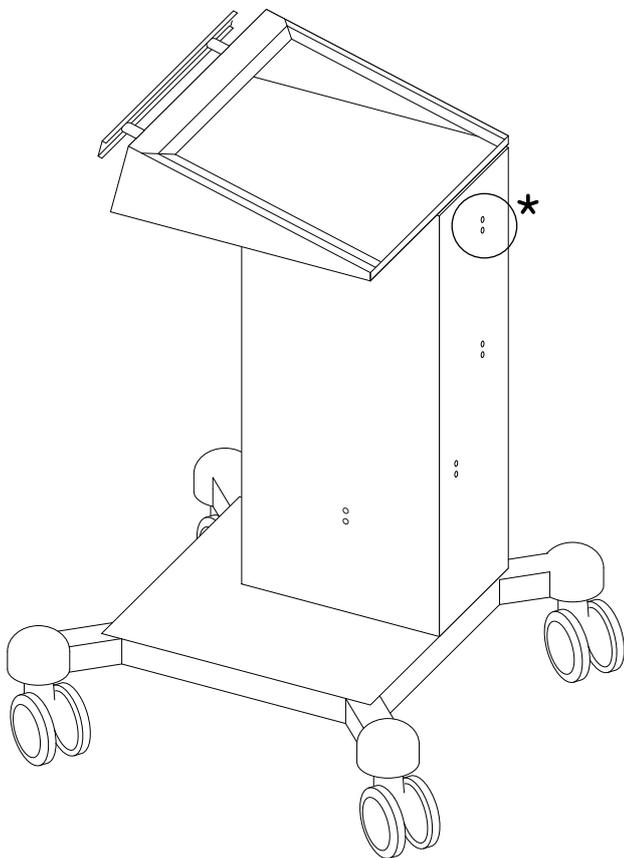
---

Permitted installation points

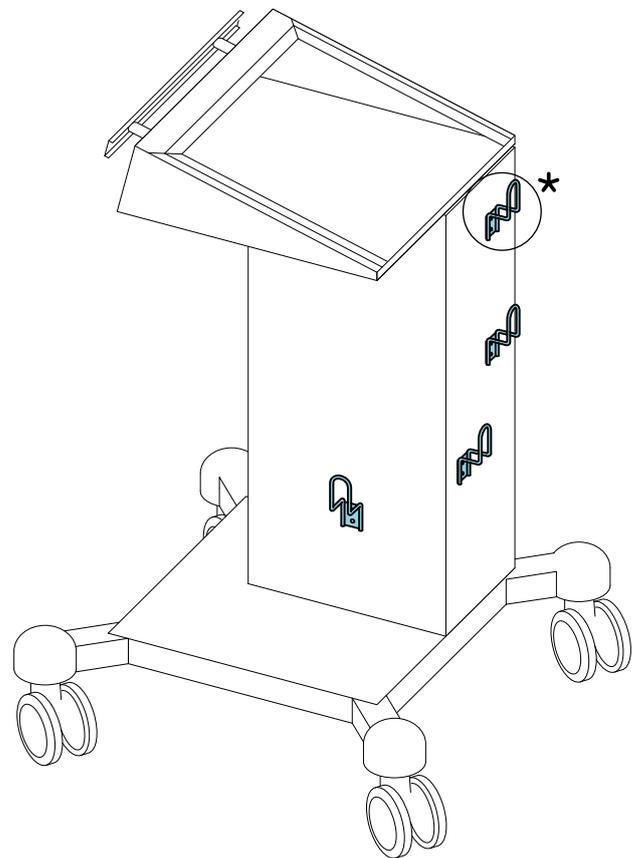Maximum loading:  
For compact, 4 items  
for compact plus, 3 items

★ Omitted for compact plus


---

## Scope of delivery

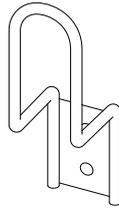---

**IMPORTANT!** Only use original Erbe accessories.

Erbe REF 20180-460




---

## Tools required

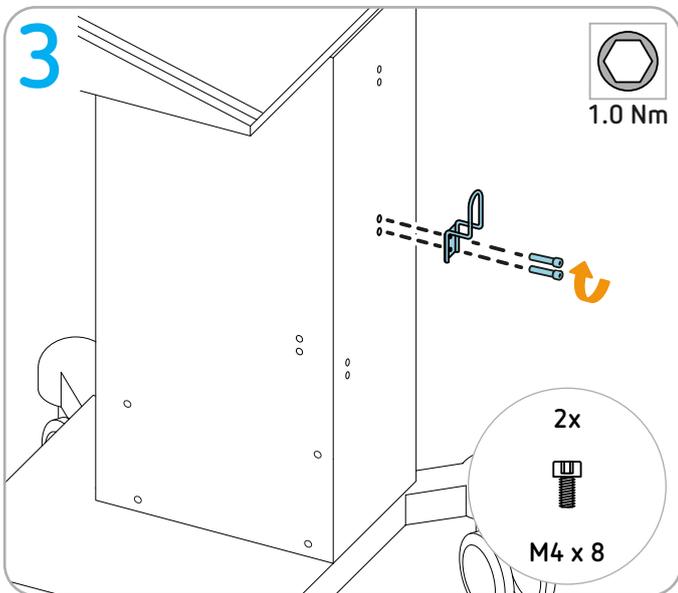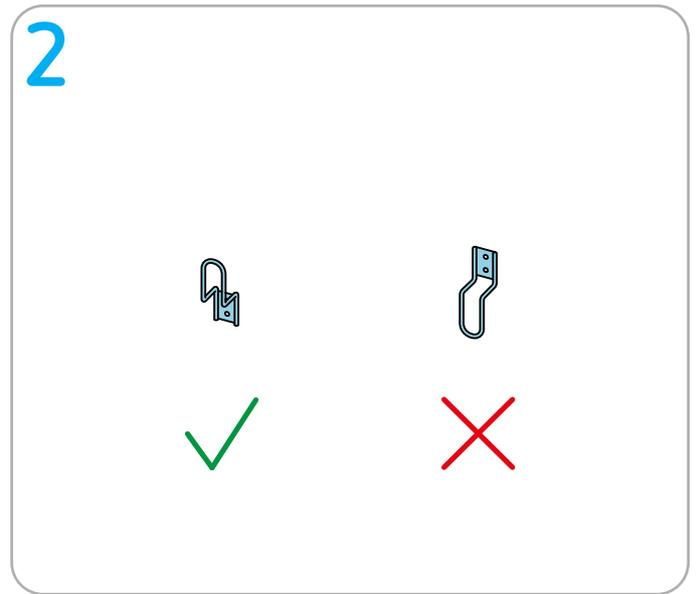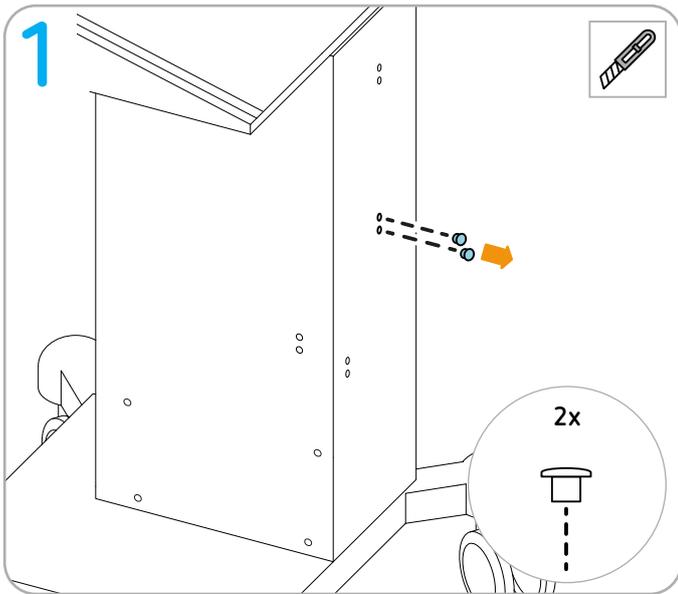---

 Torque screwdriver (1 Nm) hex socket 2.5

 Cutter


# Installing hook for cable storage

---


## Normal use

---

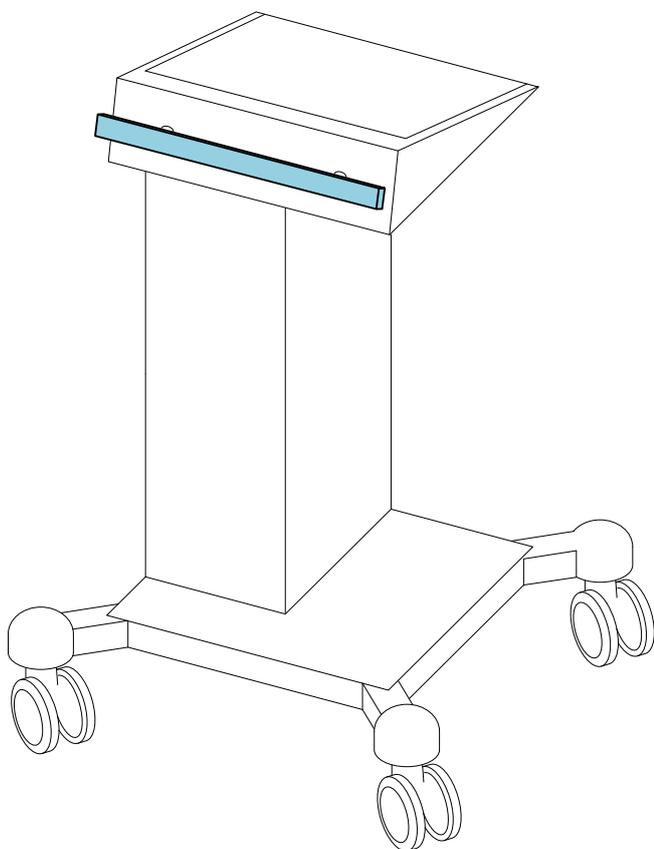Permitted attachment marked in blue

Maximum loading:  
1 item

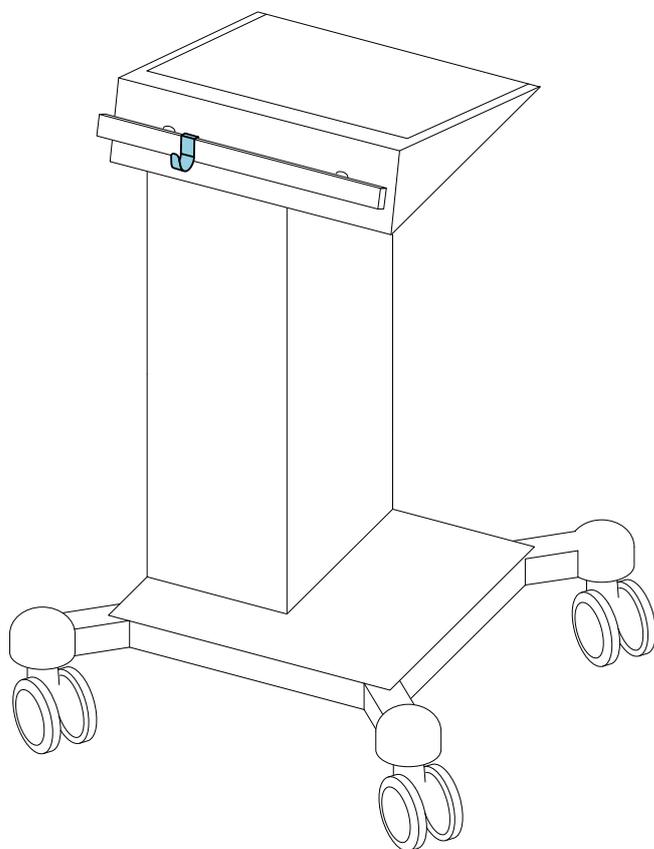
---

## Scope of delivery

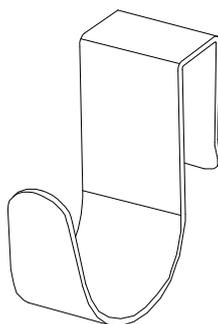---

**IMPORTANT!** Only use original Erbe accessories.

Erbe REF 20180-465



---

## Tools required

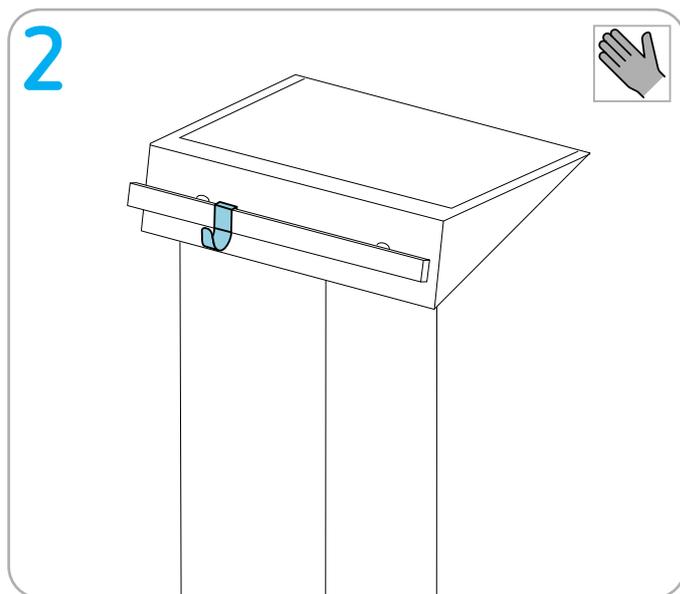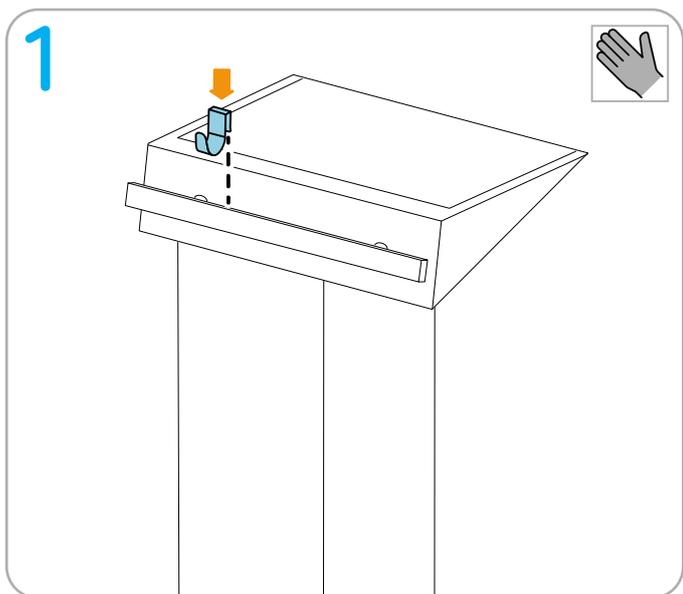---

No tool needed.


# Installing accessory basket

---


## Normal use

---

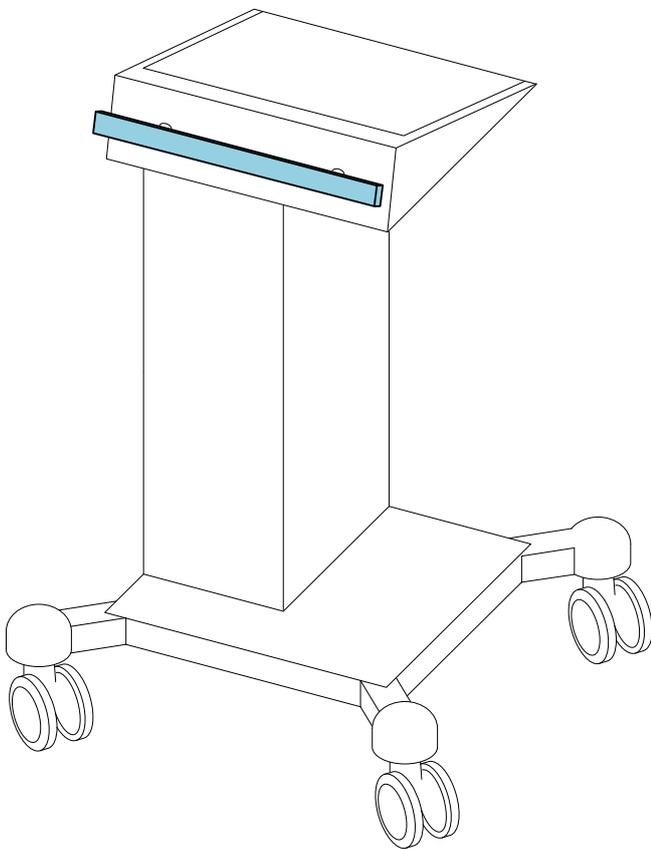Permitted attachment marked in blue

Maximum loading:  
1 item

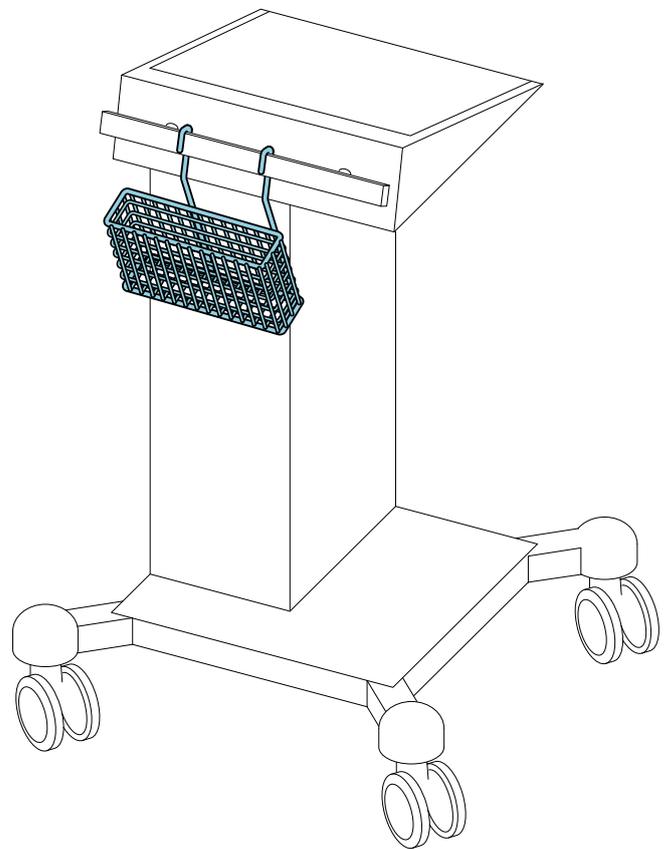
---

## Scope of delivery

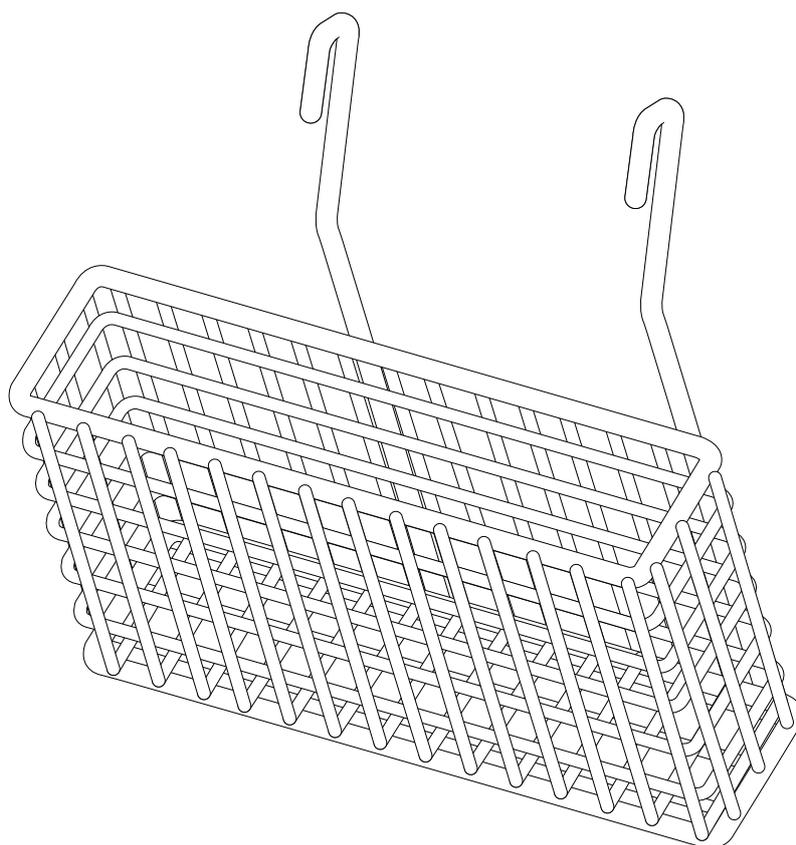---

**IMPORTANT!** Only use original Erbe accessories.

Erbe REF 20180-510



---

## Tools required

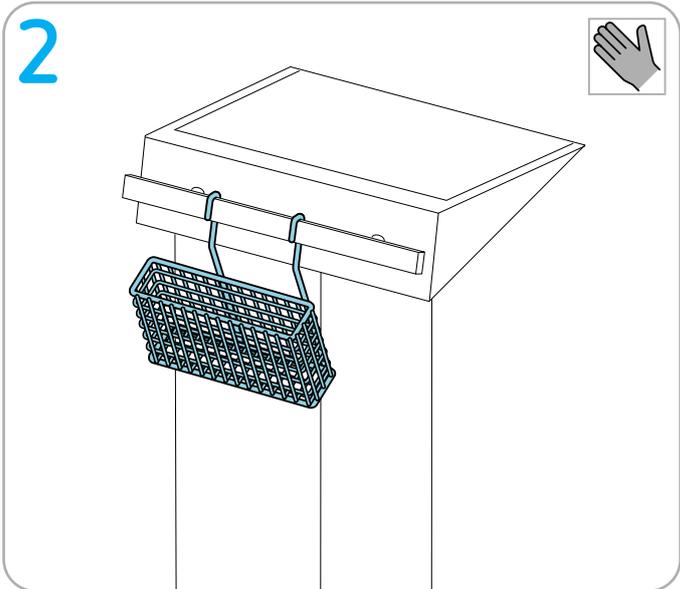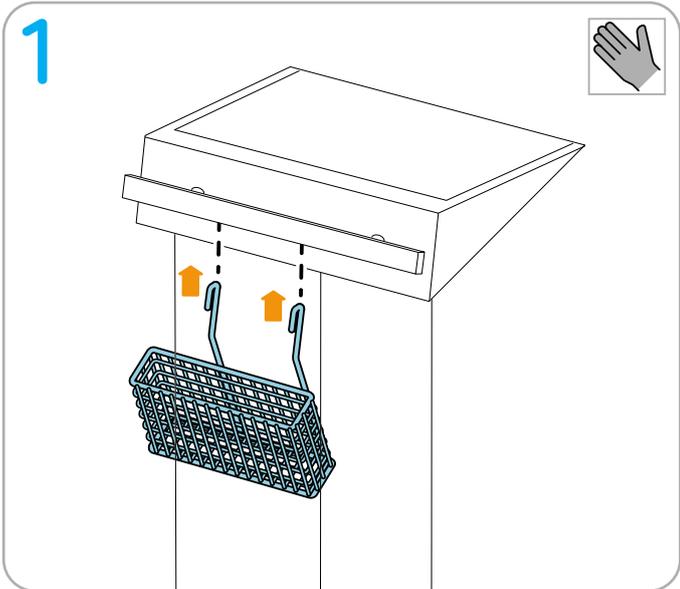---

No tool needed.

 For eIFU:  
Back to Overview

# Installing footswitch support for large one-pedal footswitch

---

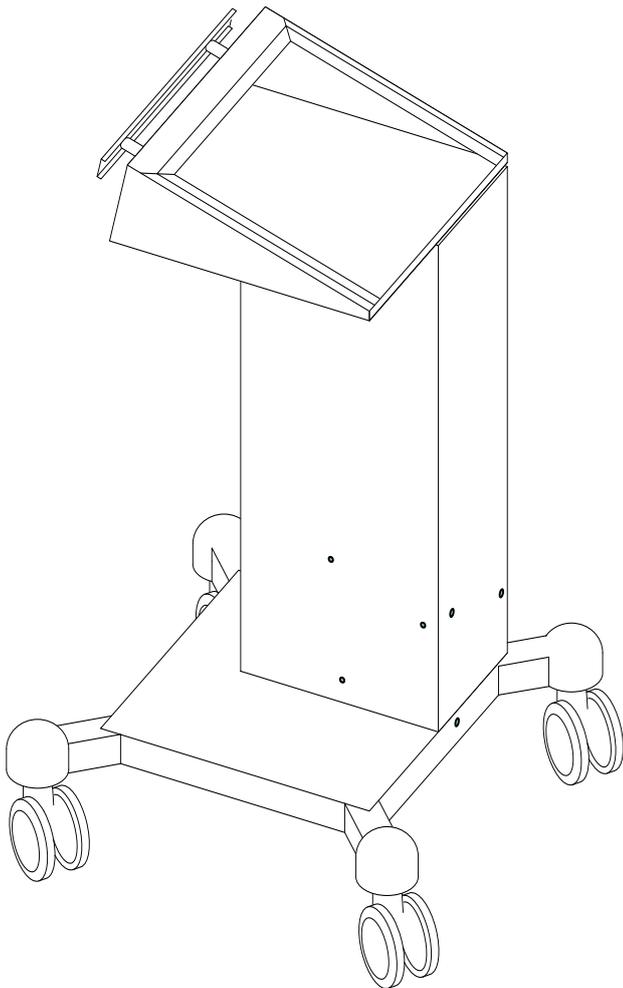
## Normal use

---


This footswitch support is compatible with the following Erbe one-pedal footswitches:

- VIO S, VIO D and VIO 3 one-pedal footswitch
- IES 3 one-pedal footswitch

Permitted installation points

Maximum loading: 2 items

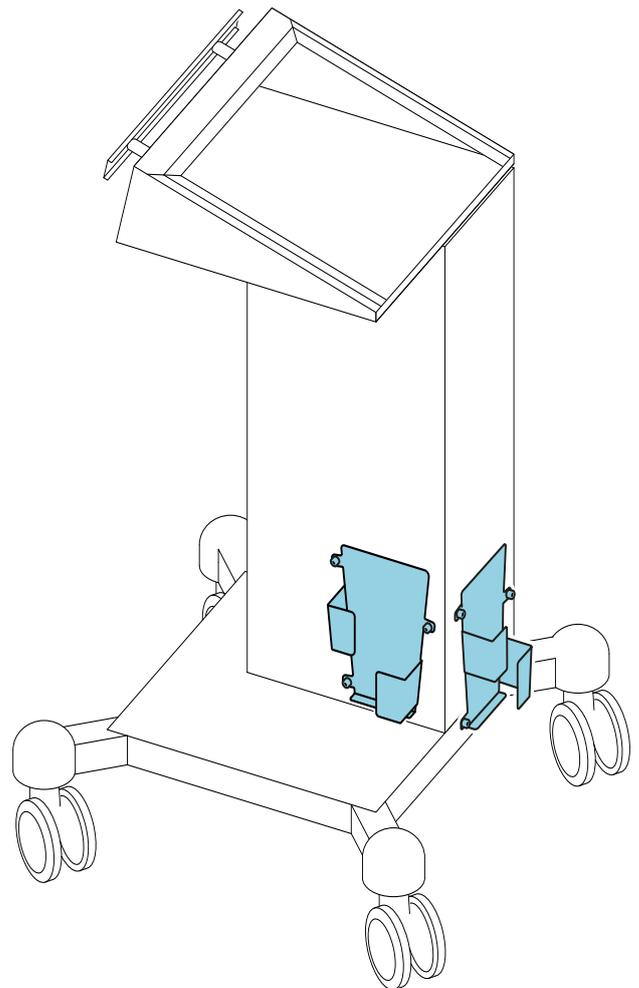
---

## Scope of delivery

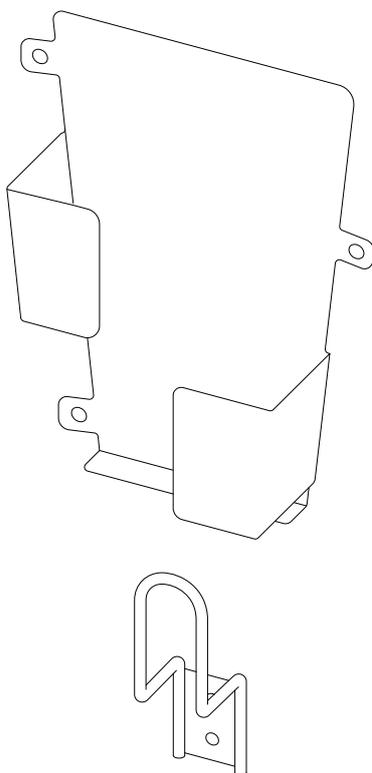---

**IMPORTANT!** Only use original Erbe accessories.

### Erbe REF 20180-463




---

## Tools required

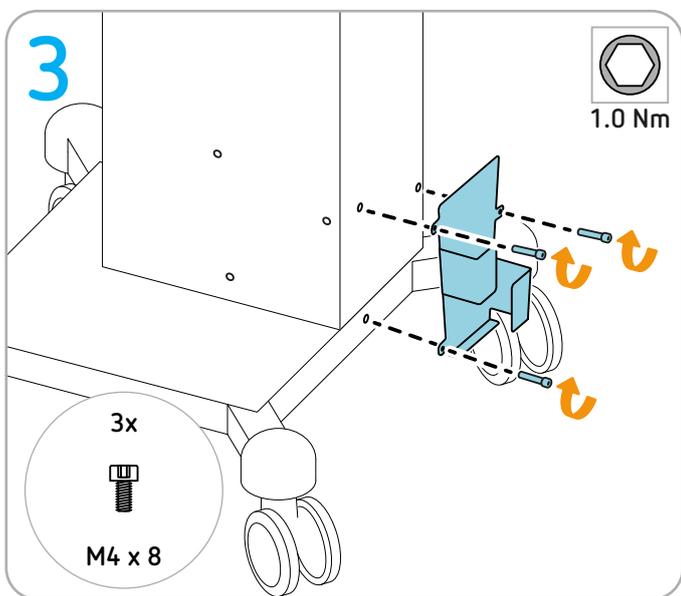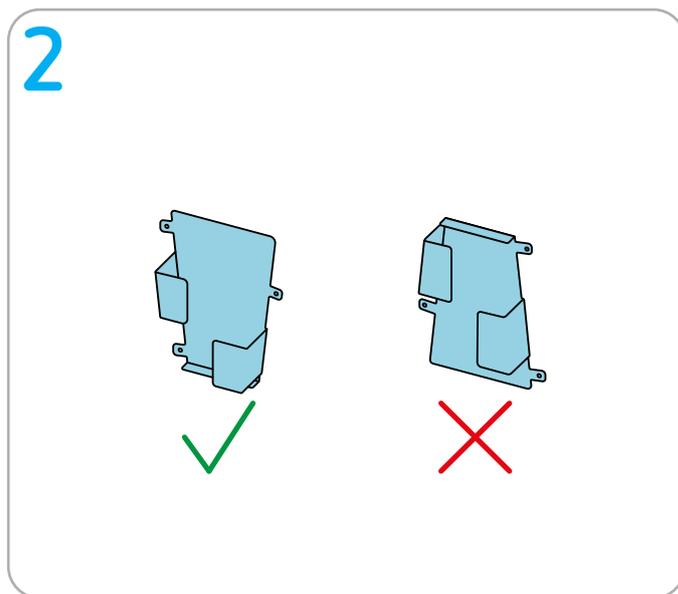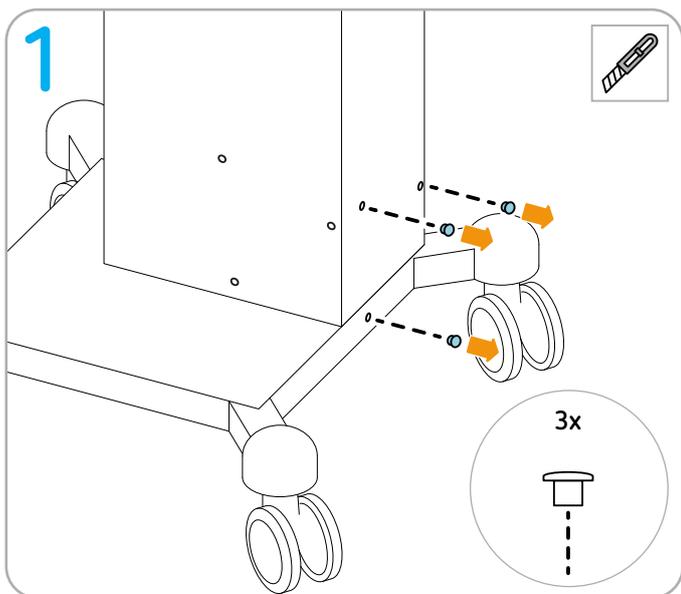---

 Torque screwdriver (1 Nm) hex socket 2.5

 Cutter

# Installing footswitch support for small one-pedal footswitch on the system carrier

---

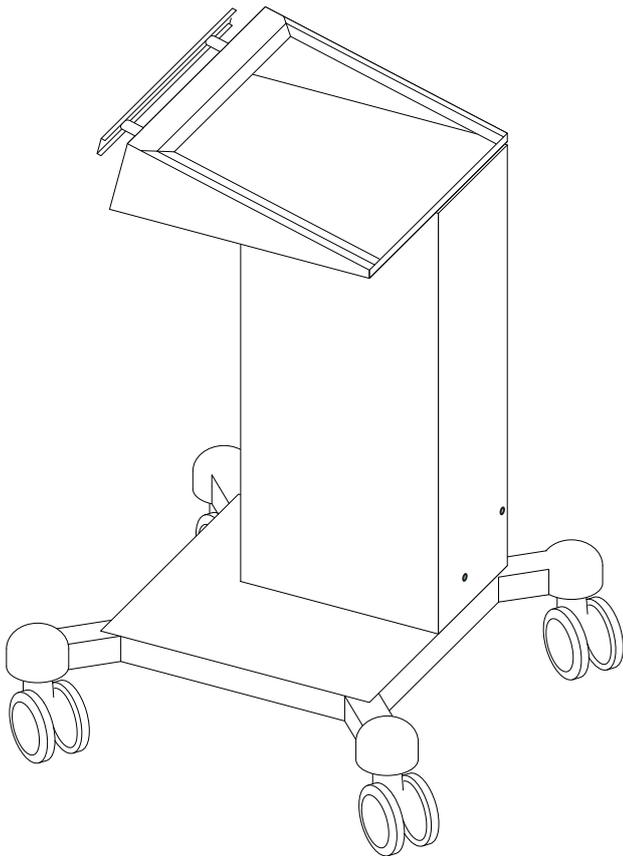
## Normal use

---


This footswitch support is compatible with the following Erbe one-pedal footswitches:

- VIO C one-pedal footswitch
- EIP 2 one-pedal footswitch

Permitted installation points

Maximum loading: 1 item



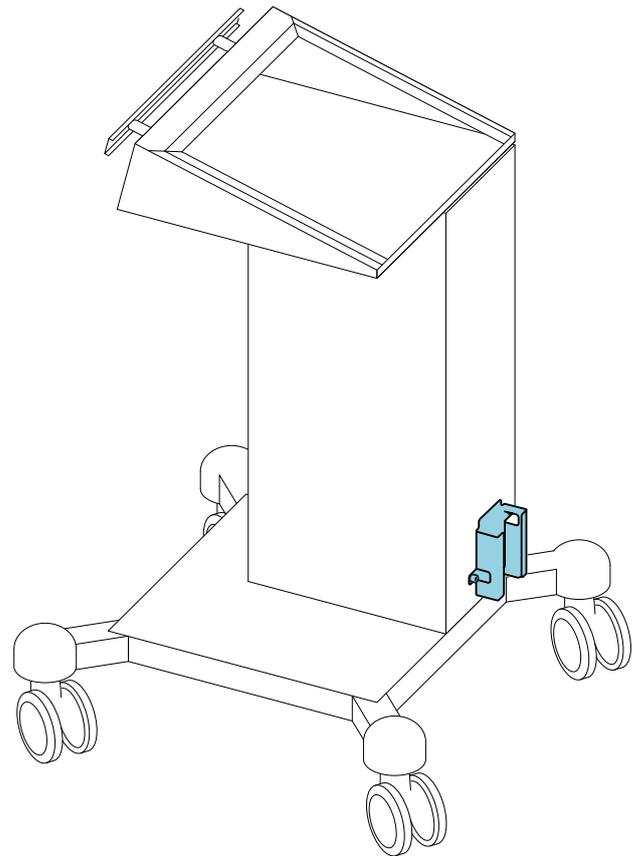
---

## Scope of delivery

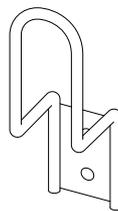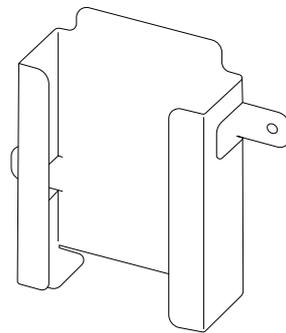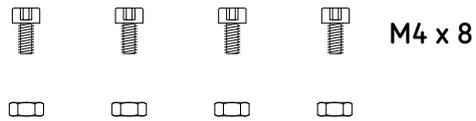---

**IMPORTANT!** Only use original Erbe accessories.

### Erbe REF 20180-464

Universal fastening set: Not all parts are required for the following assembly steps.




---

## Tools required

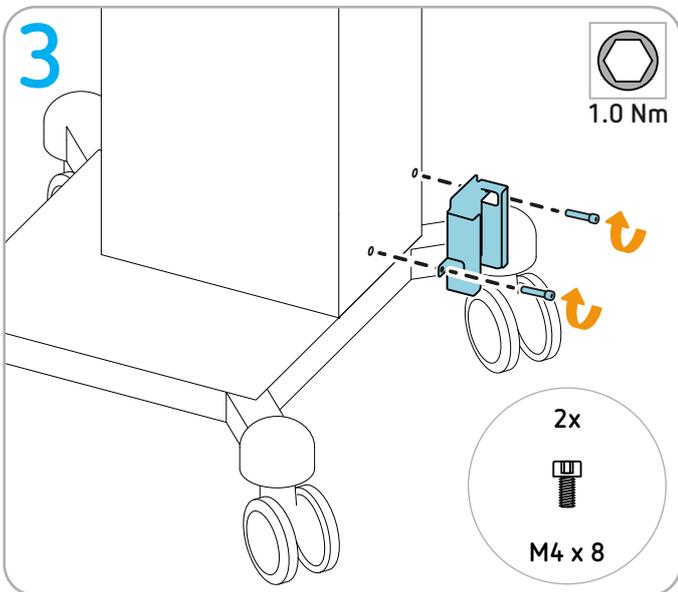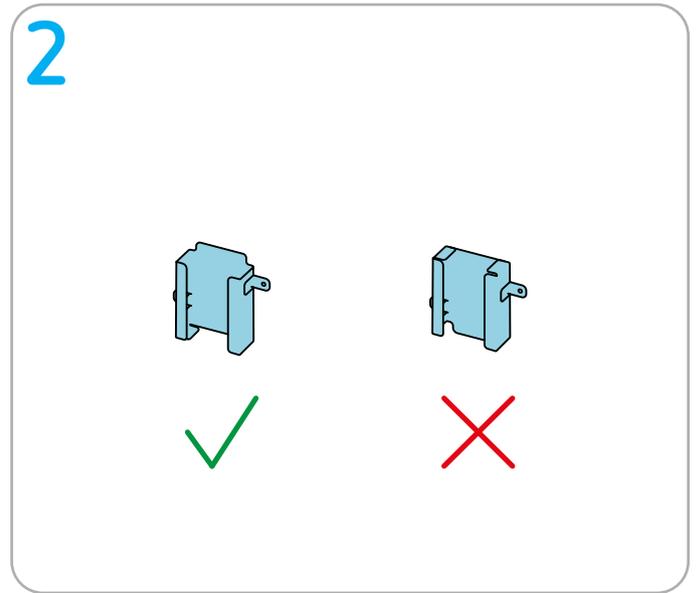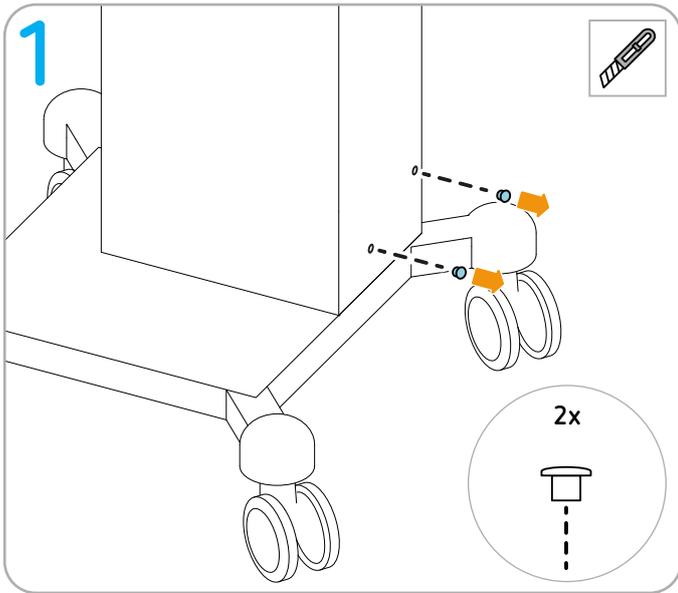---

 Torque screwdriver (1 Nm) hex socket 2.5

 Cutter


# Installing IES 3

---


## Permitted positioning

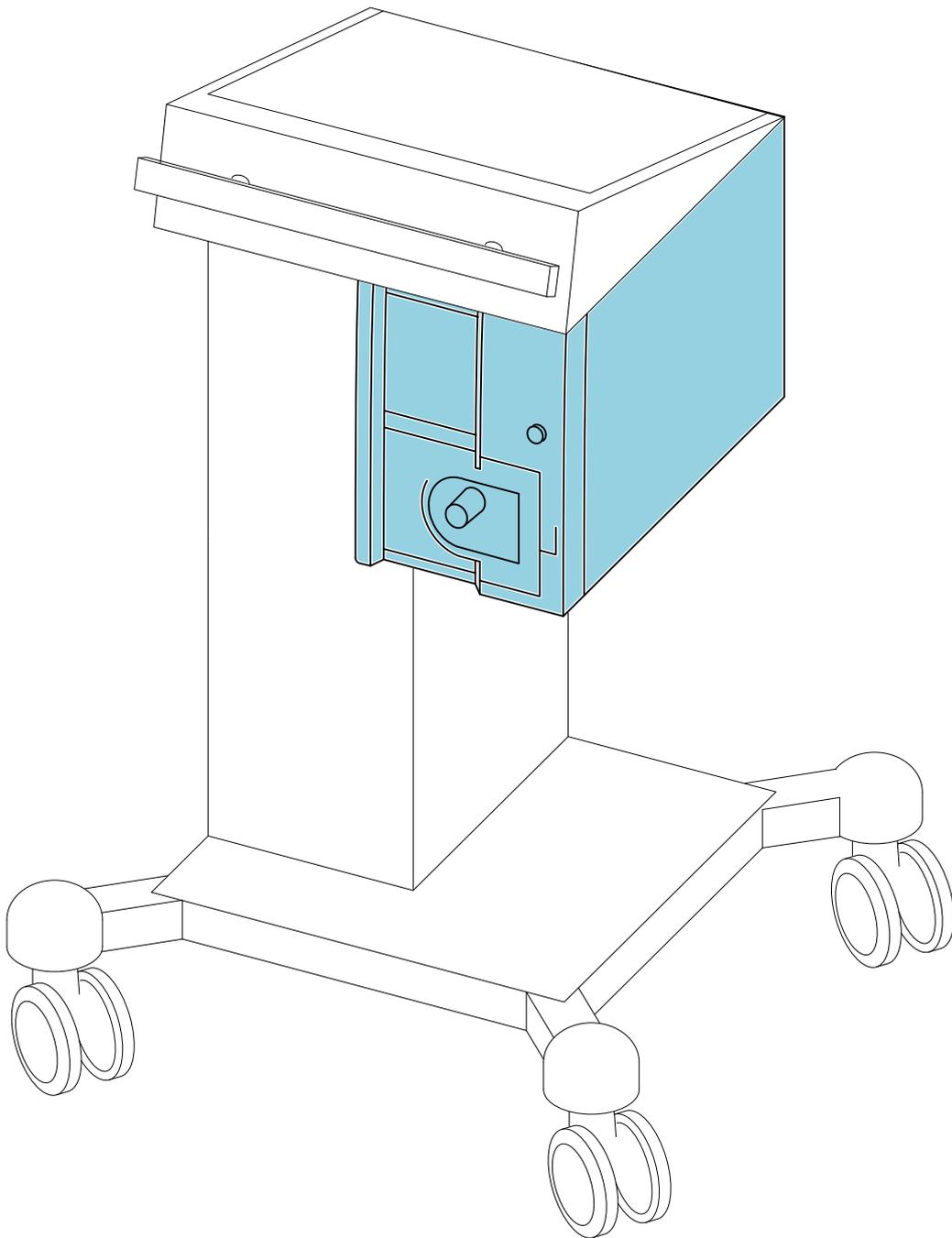---

## Erbe fastening set

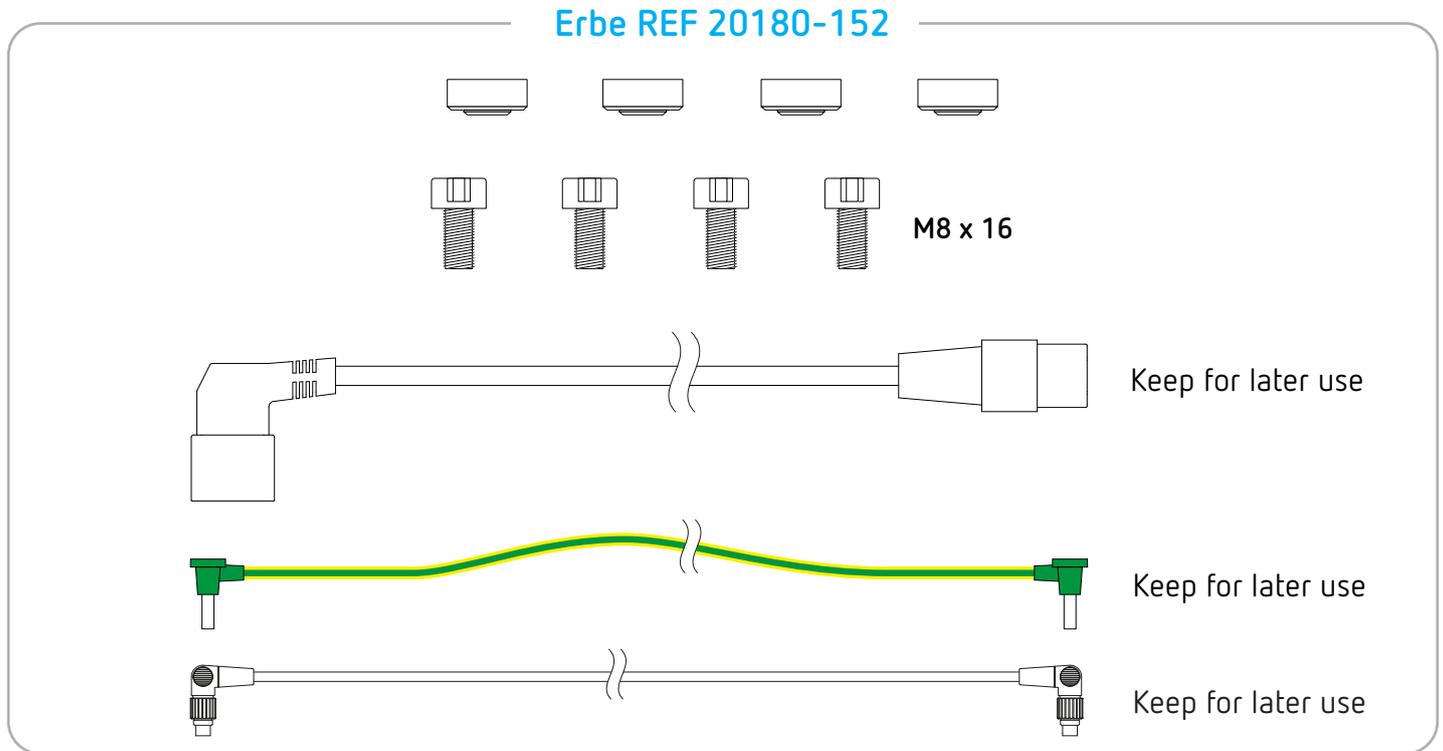**IMPORTANT!** Only use original Erbe accessories.







## Tools required

-  Torque screwdriver (1,5 Nm) hex socket 6
-  Torque screwdriver (1,5 Nm) hex socket 2,5
-  Cutter

## Preparation

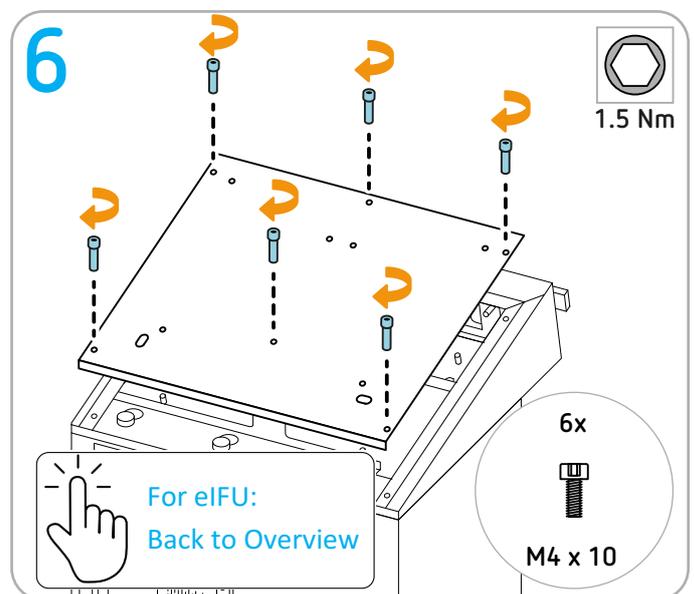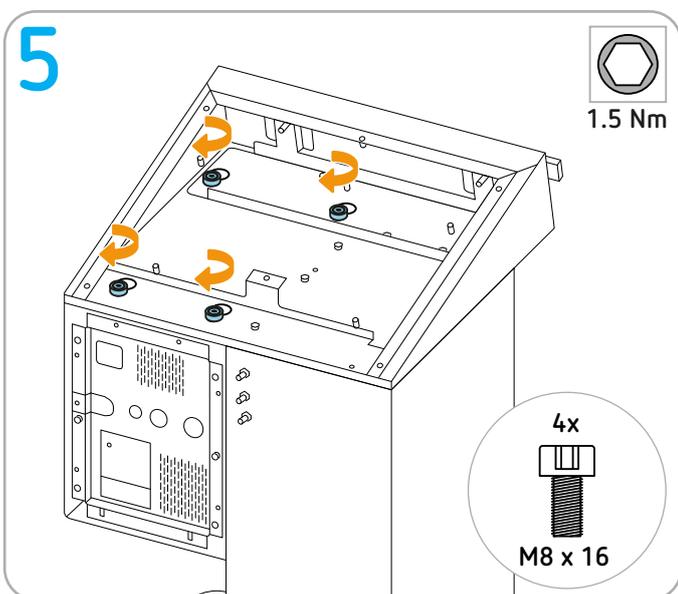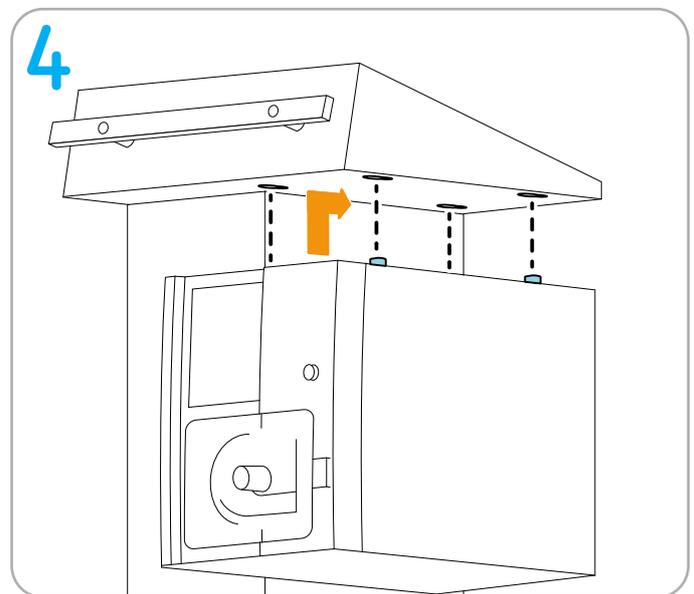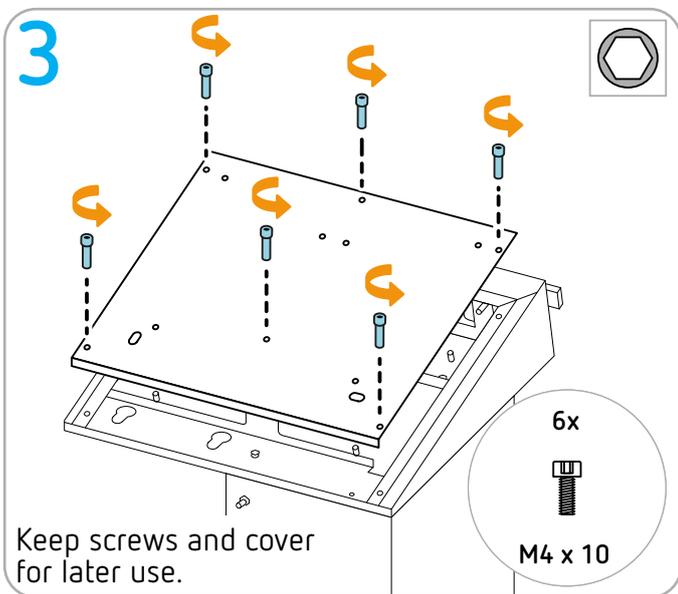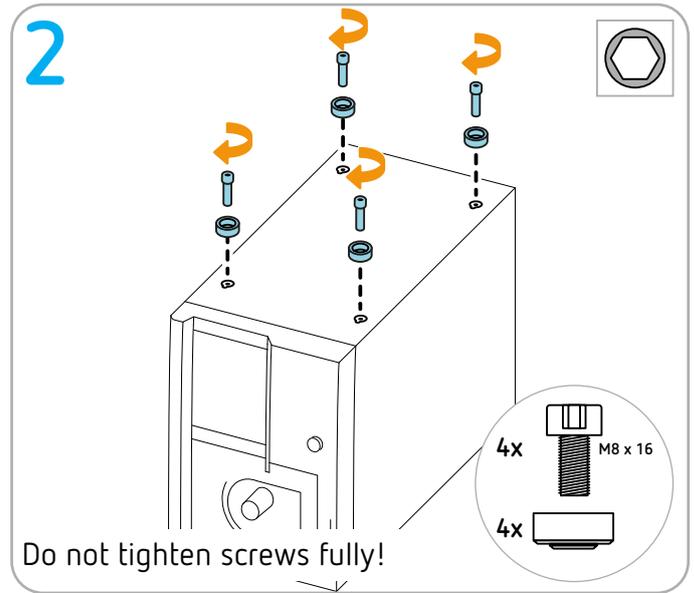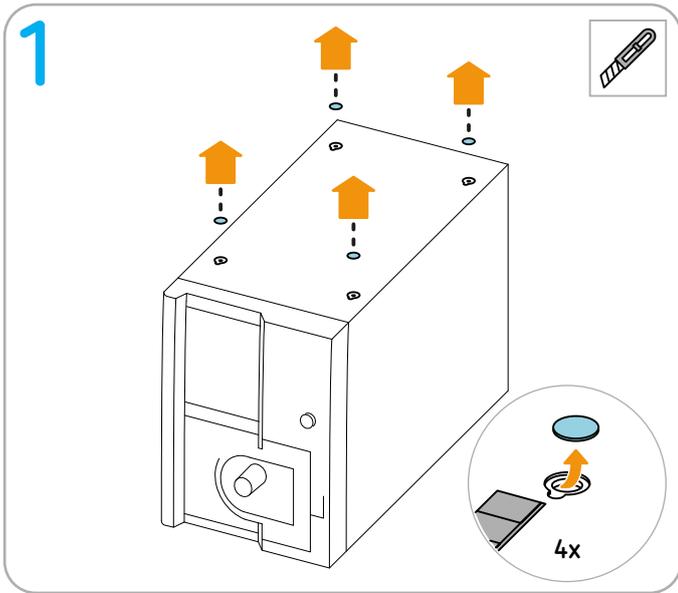**IMPORTANT!** Before installing the units on the system carrier, remove all cables, tubing, and instruments.


# Installing EIP 2

---

## Permitted positioning

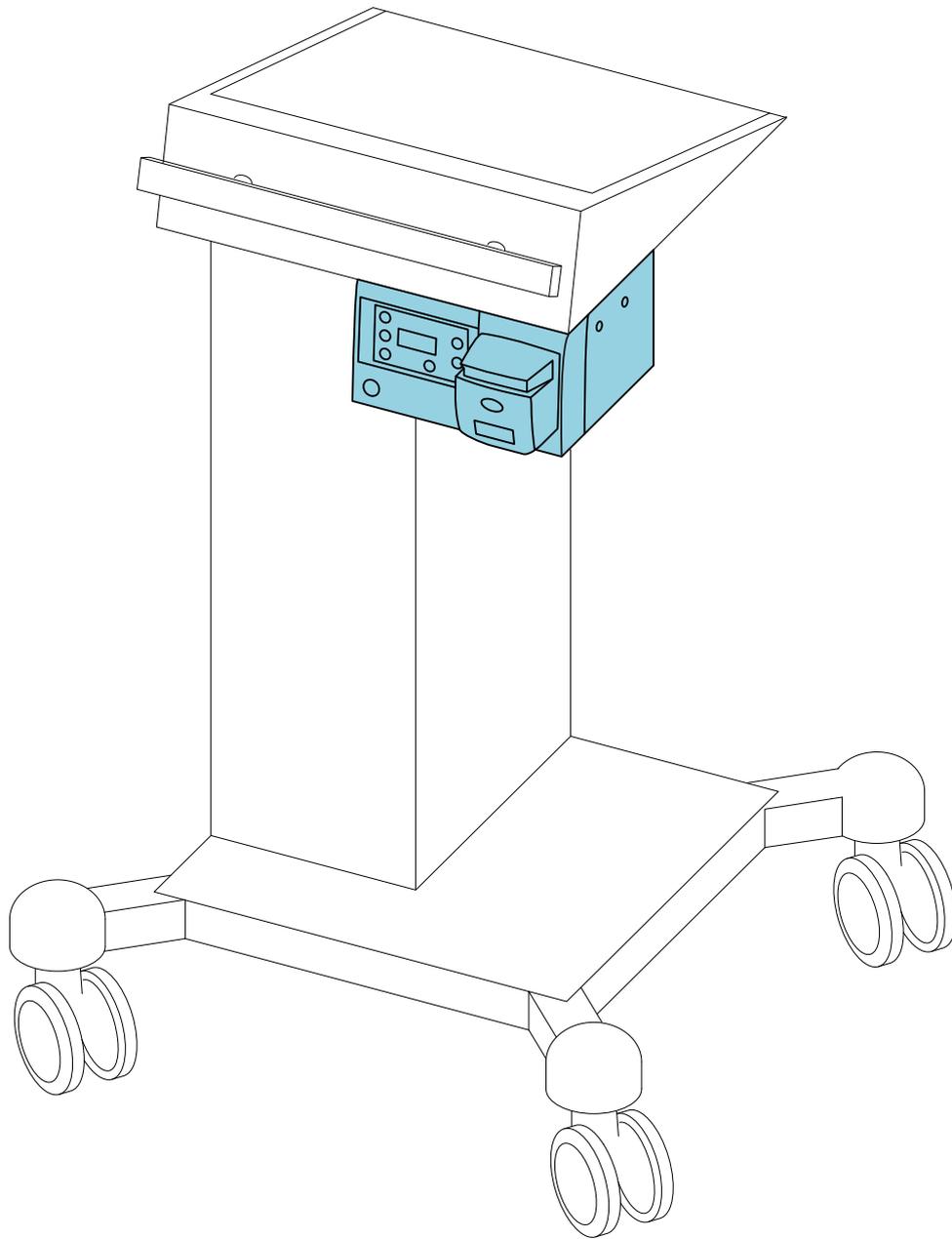---


---

## Erbe fastening set

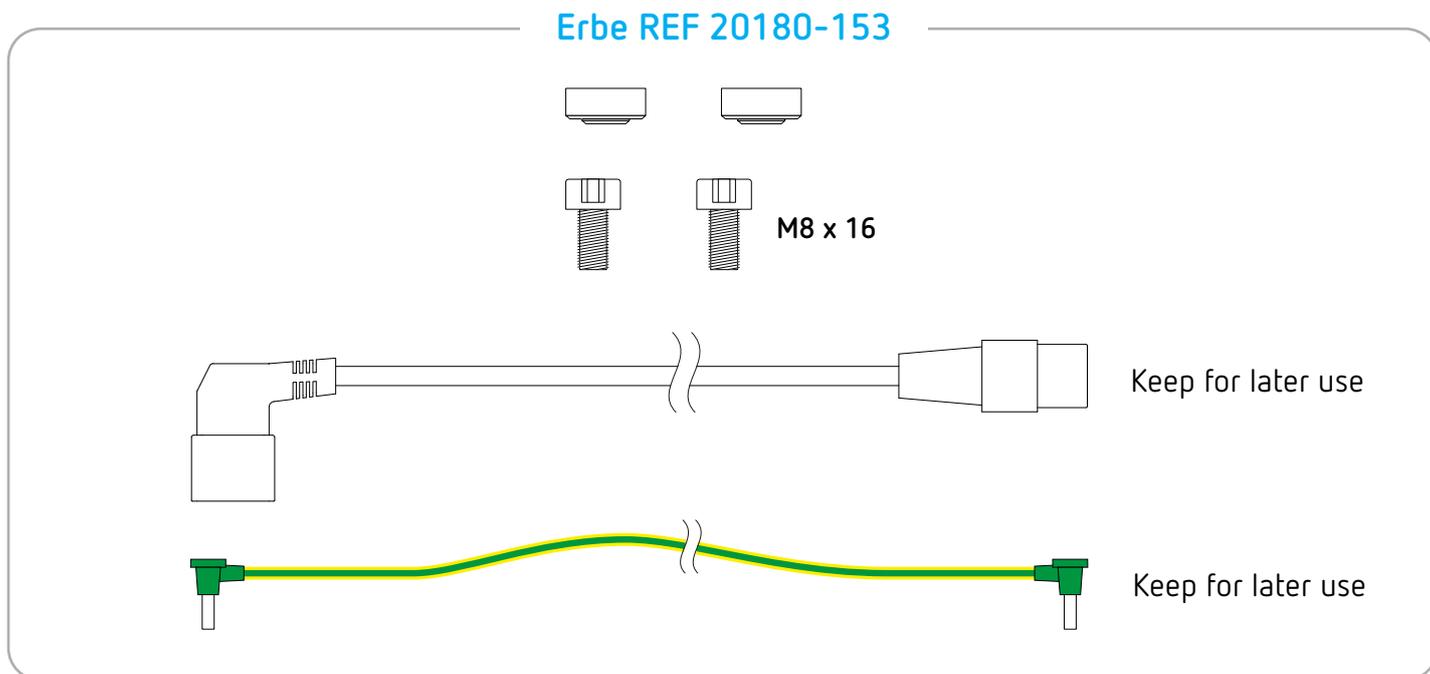---

**IMPORTANT!** Only use original Erbe accessories.

---

## Tools required

---

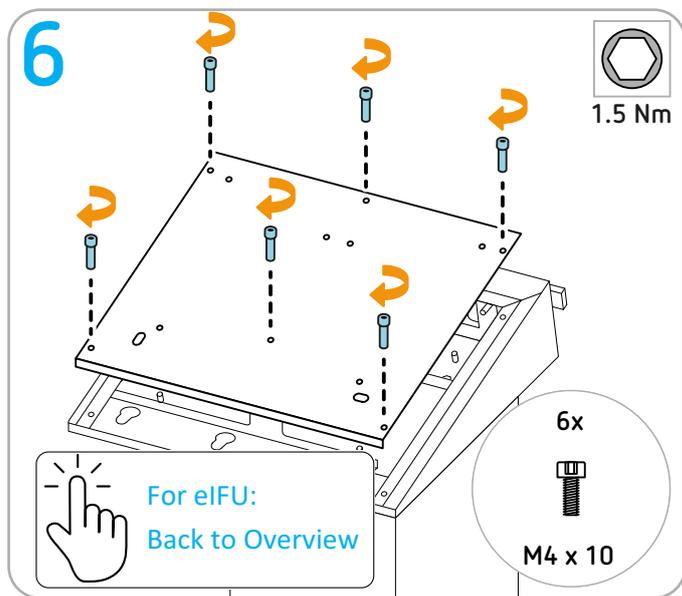
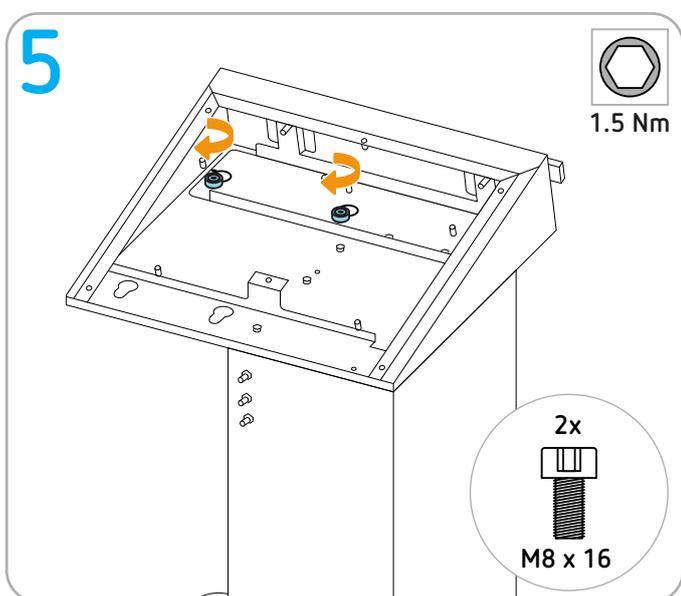
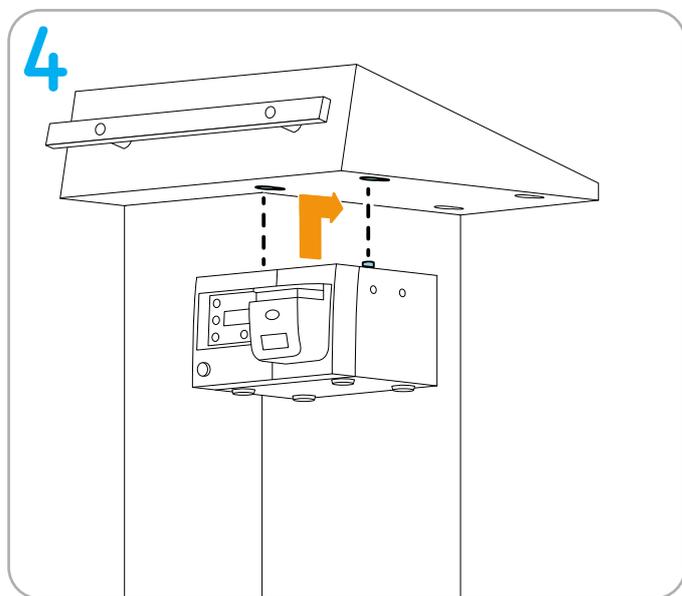
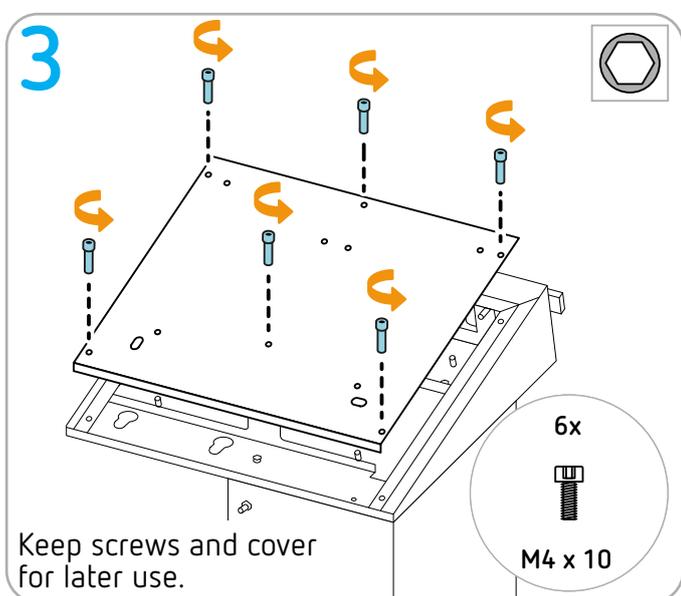
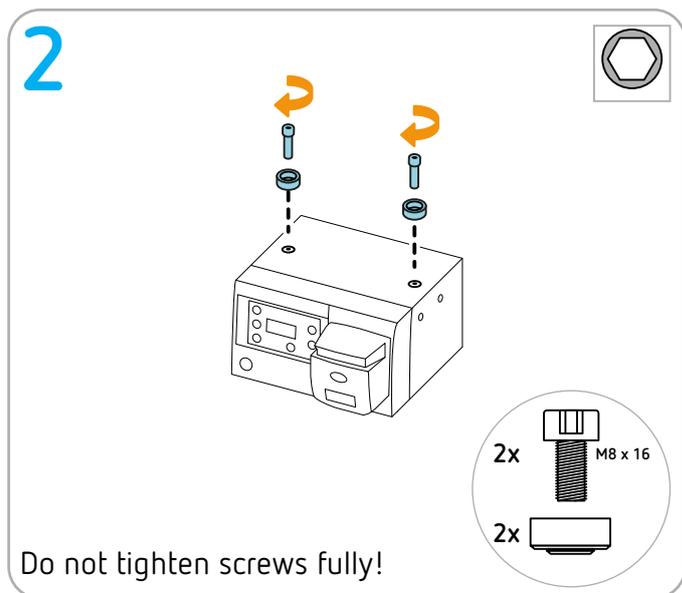
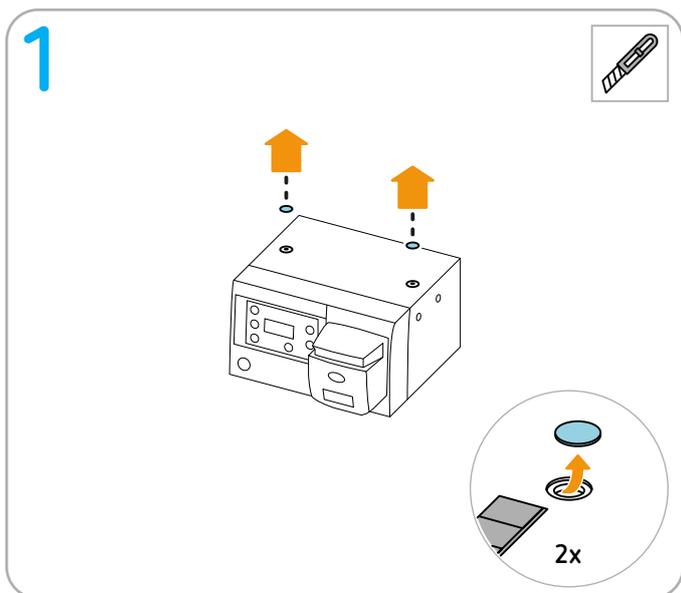
-  Torque screwdriver (1,5 Nm) hex socket 6
-  Torque screwdriver (1,5 Nm) hex socket 2,5
-  Cutter

---

## Preparation

---

**IMPORTANT!** Before installing the units on the system carrier, remove all cables, tubing, and instruments.


# Installing VIO C series

---

## Permitted positioning

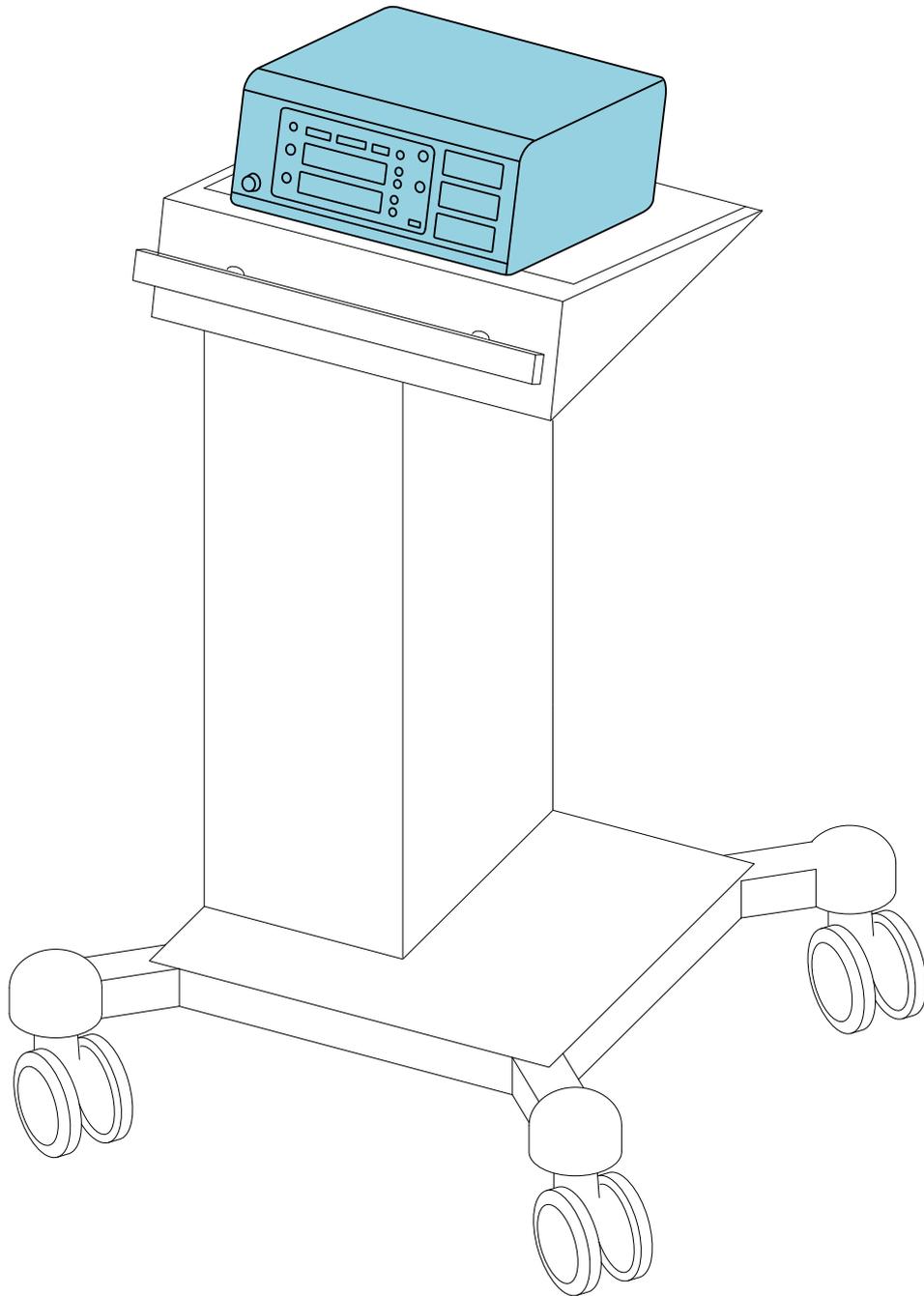---


---

## Erbe fastening set

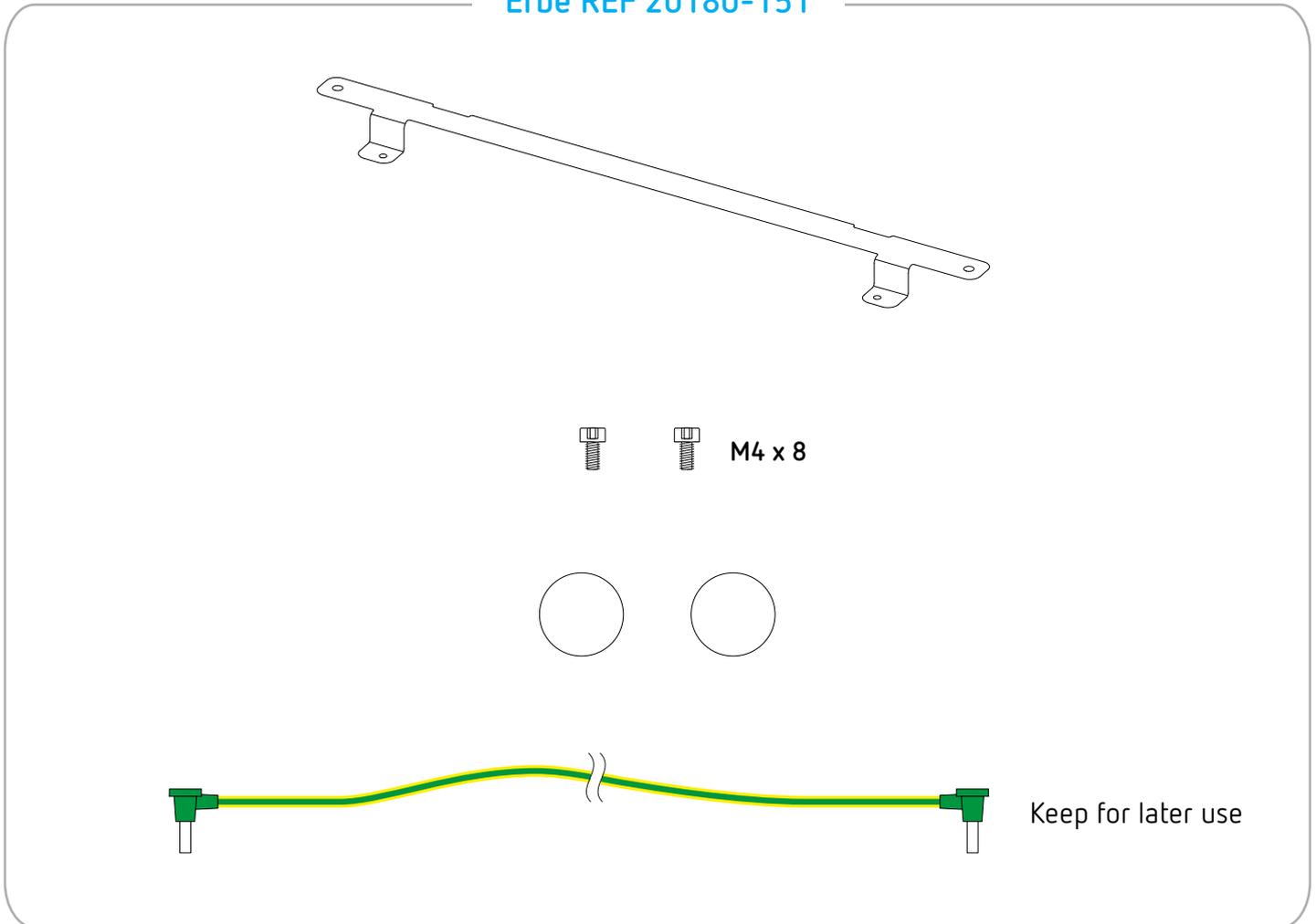---

**IMPORTANT!** Only use original Erbe accessories.

Erbe REF 20180-151

---

## Tools required

---

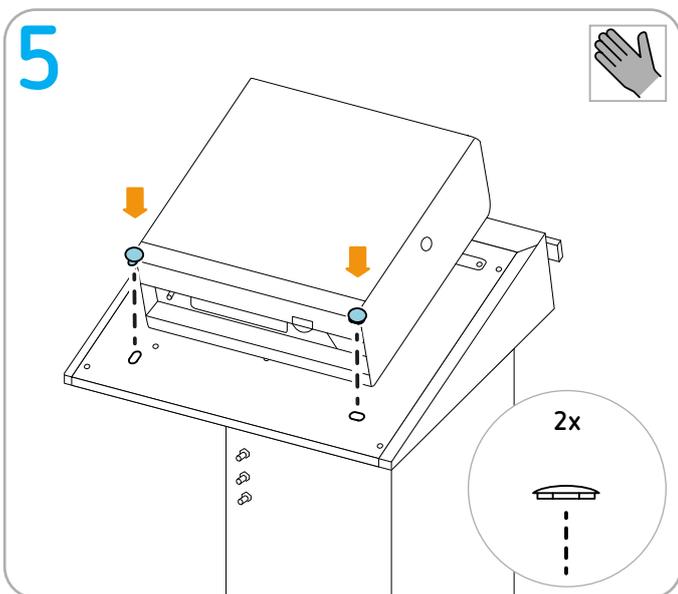
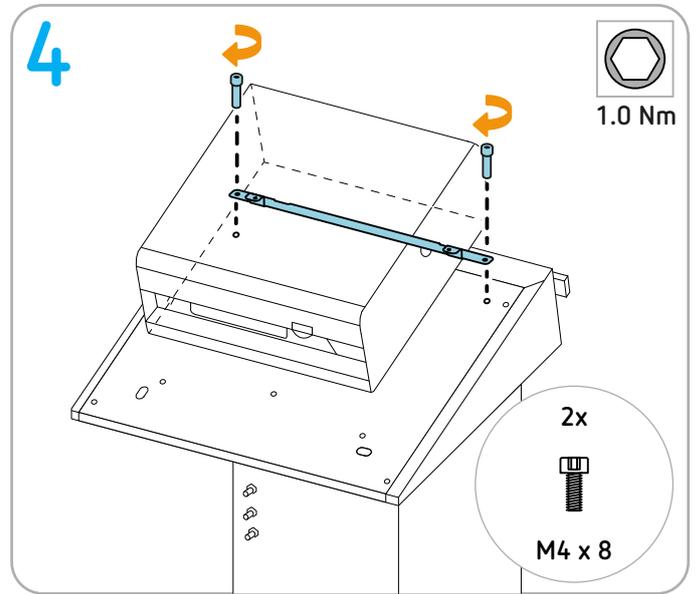
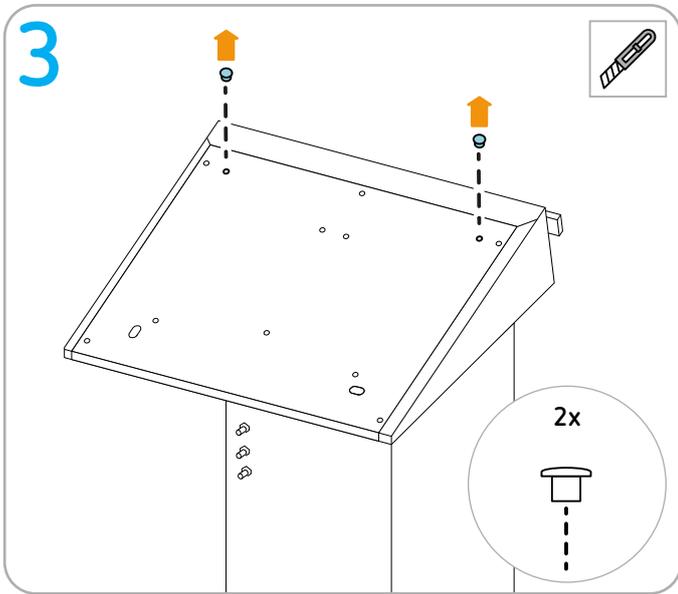
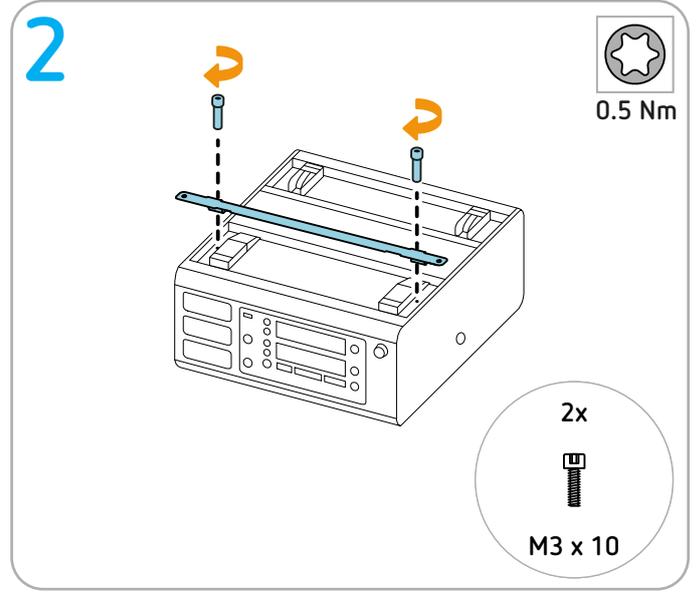
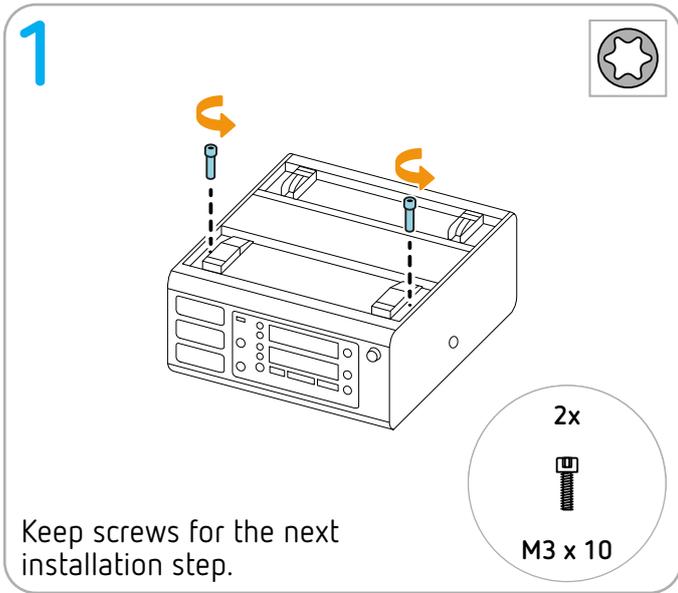
-  Torque screwdriver (1 Nm) hex socket 2.5
-  Torque screwdriver (0.5 Nm) Torx 10
-  Cutter

---

## Preparation

---

**IMPORTANT!** Before installing the units on the system carrier, remove all cables, tubing, and instruments.


# Installing VIO S, VIO D or VIO 3 series

---


## Permitted positioning

---

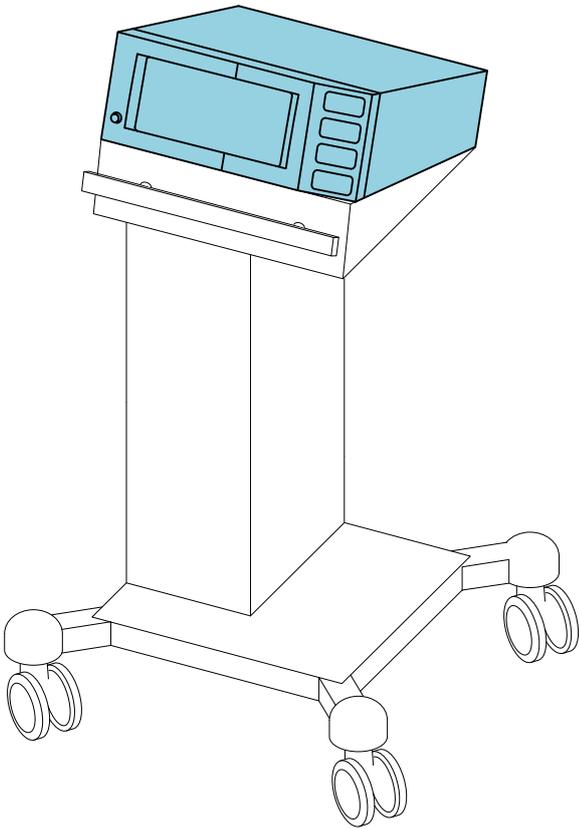VIO S series and  
VIO D series

VIO 3 and  
VIO 3n
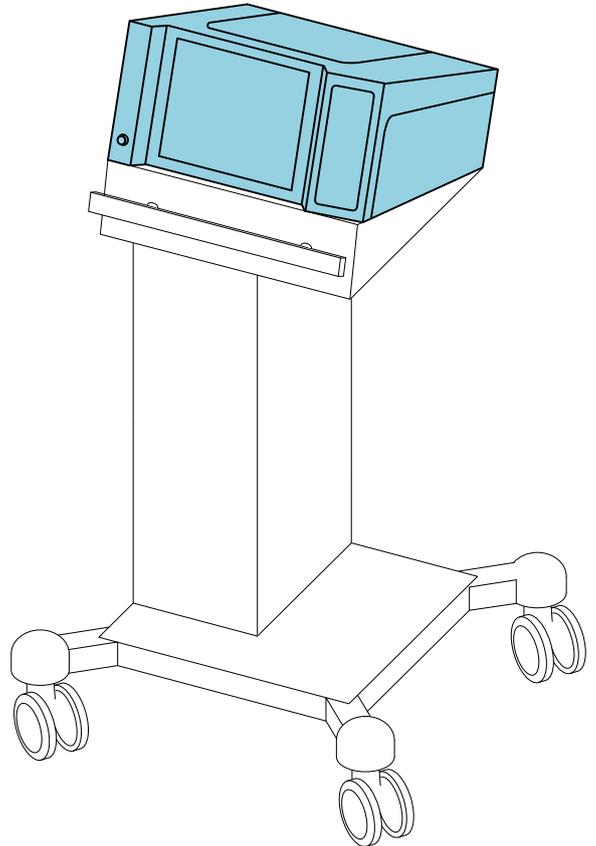
## Erbe fastening set

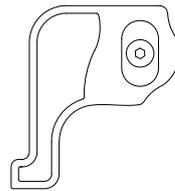
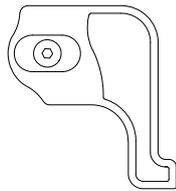
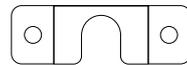
**IMPORTANT!** Only use original Erbe accessories.

**IMPORTANT!** Order the protective caps for the via of the units (see Step 1) separately as needed; they are not part of this fastening set 20180-150. Protective cap for VIO 3 via Erbe REF 40160-112 (1 item needed), protective cap for VIO S or VIO D via Erbe REF 40140-099 (2 items needed).


### Erbe REF 20180-150

M4 x 8

M4 x 16 screws included

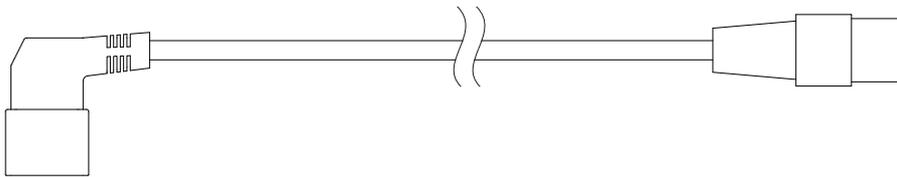Keep for later use

Keep for later use







## Tools required

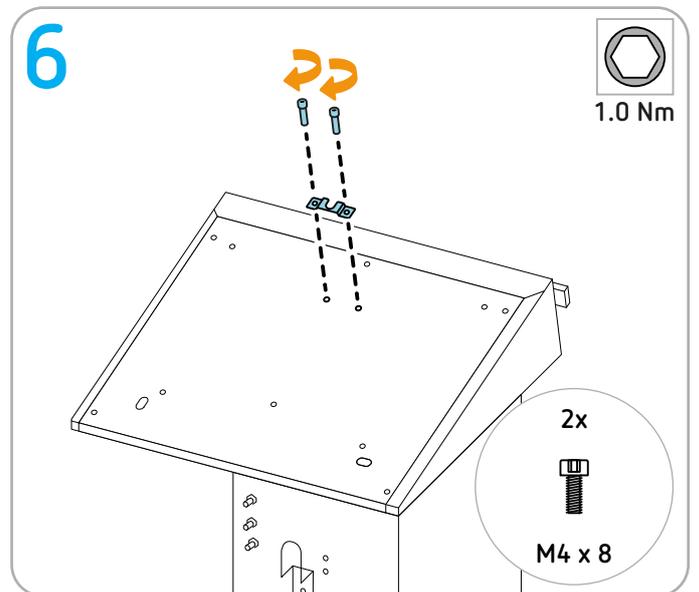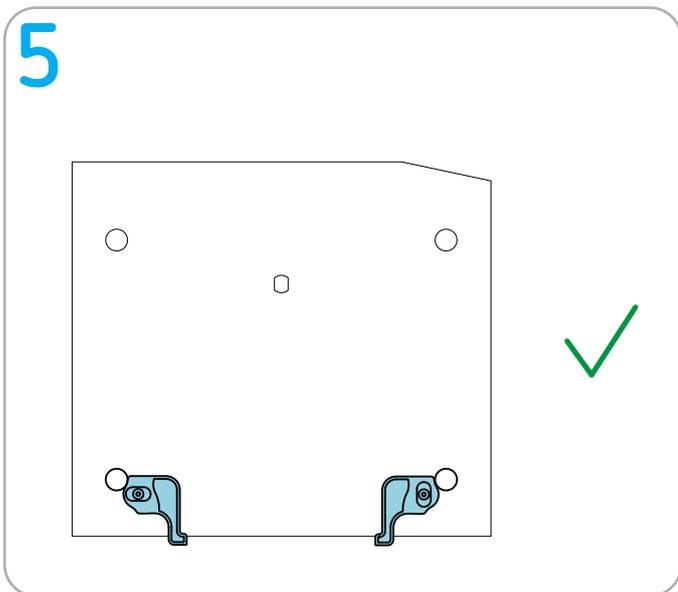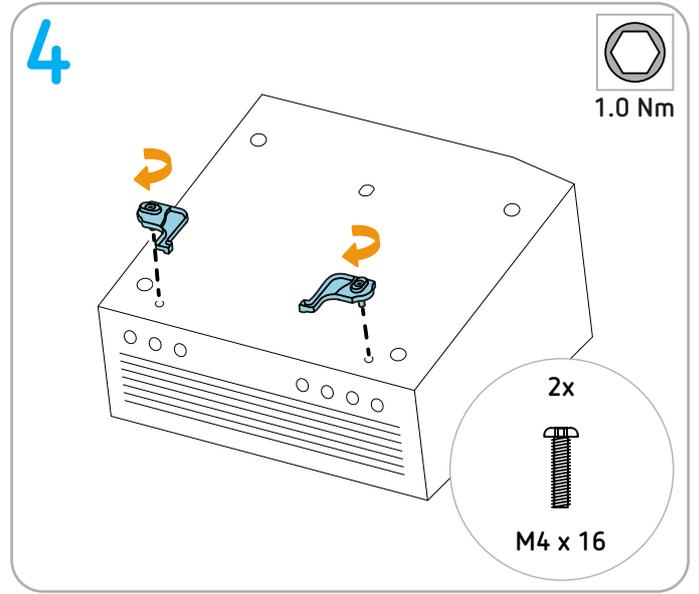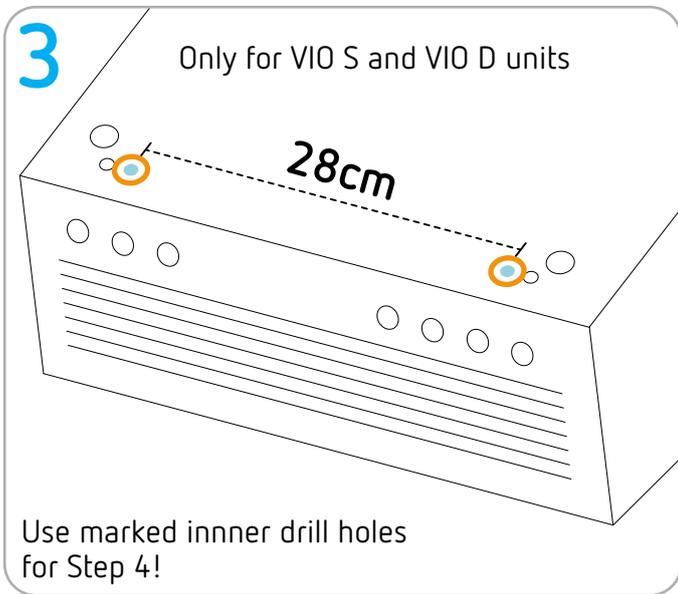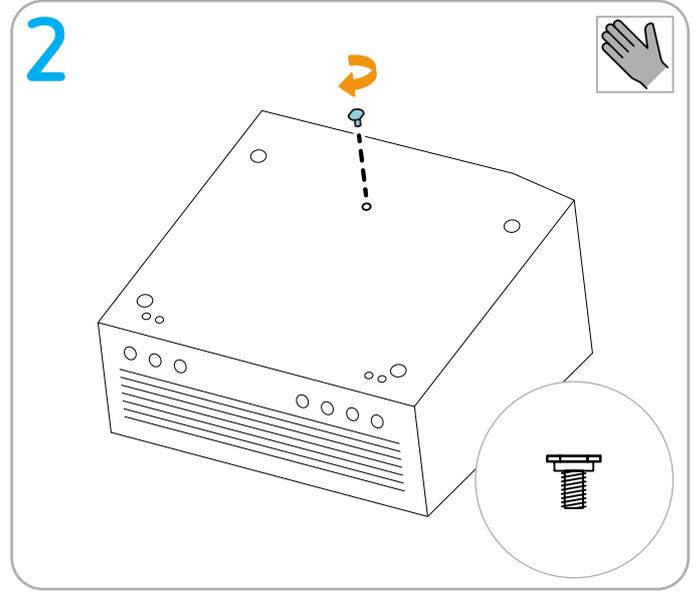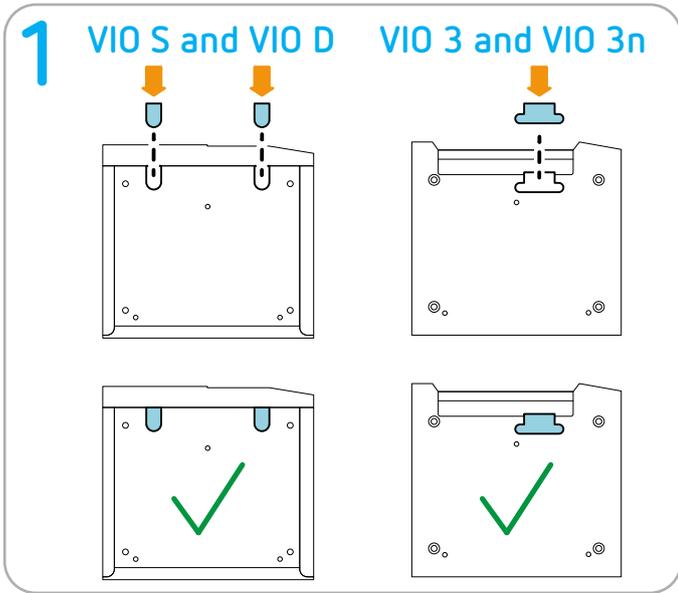
Torque screwdriver (1 Nm) hex socket 2.5

## Preparation


**IMPORTANT!** Before installing the units on the system carrier, remove all cables, tubing, and instruments.


## 7 Attachment points on the system carrier

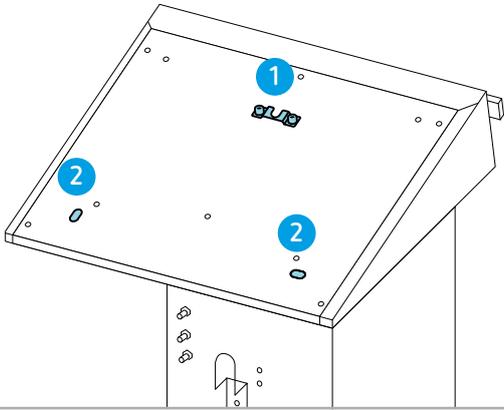
## 8 Attachment points on the unit

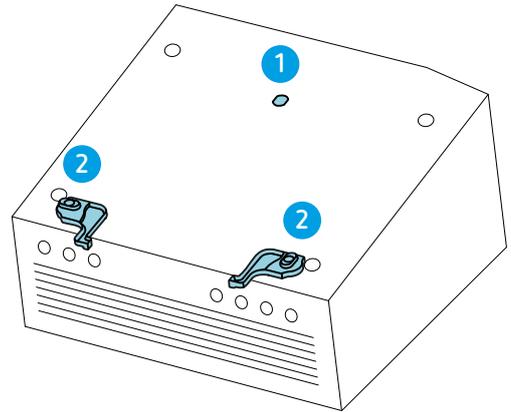
## 9
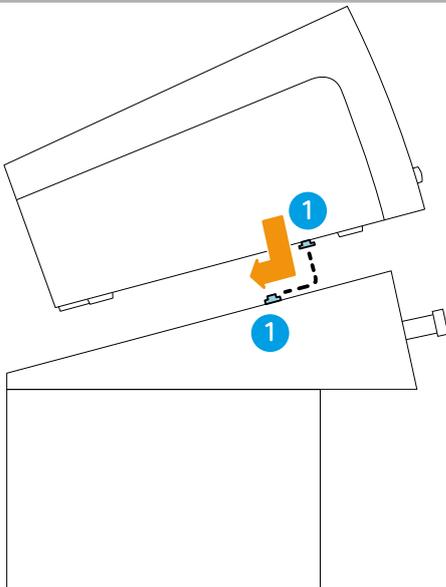
## 10
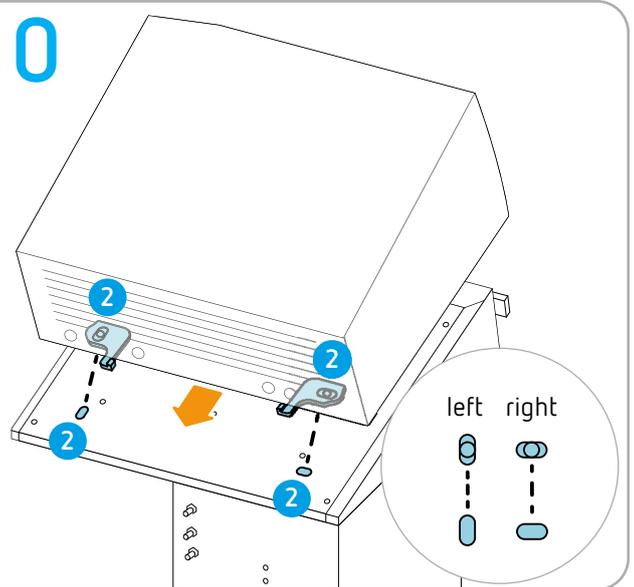
## 11

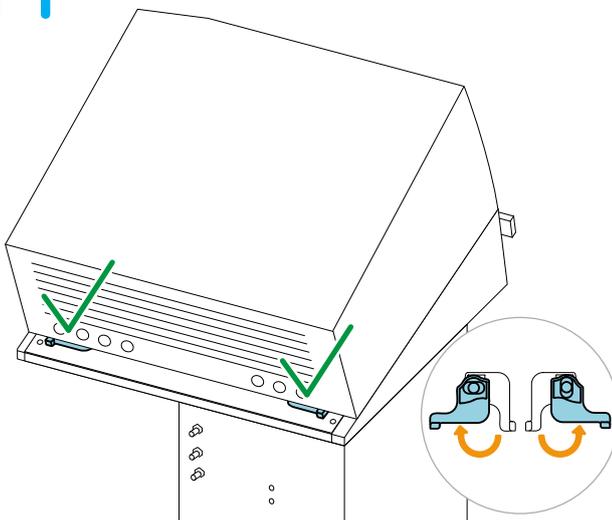
# Installing VIO seal

---

## Permitted positioning

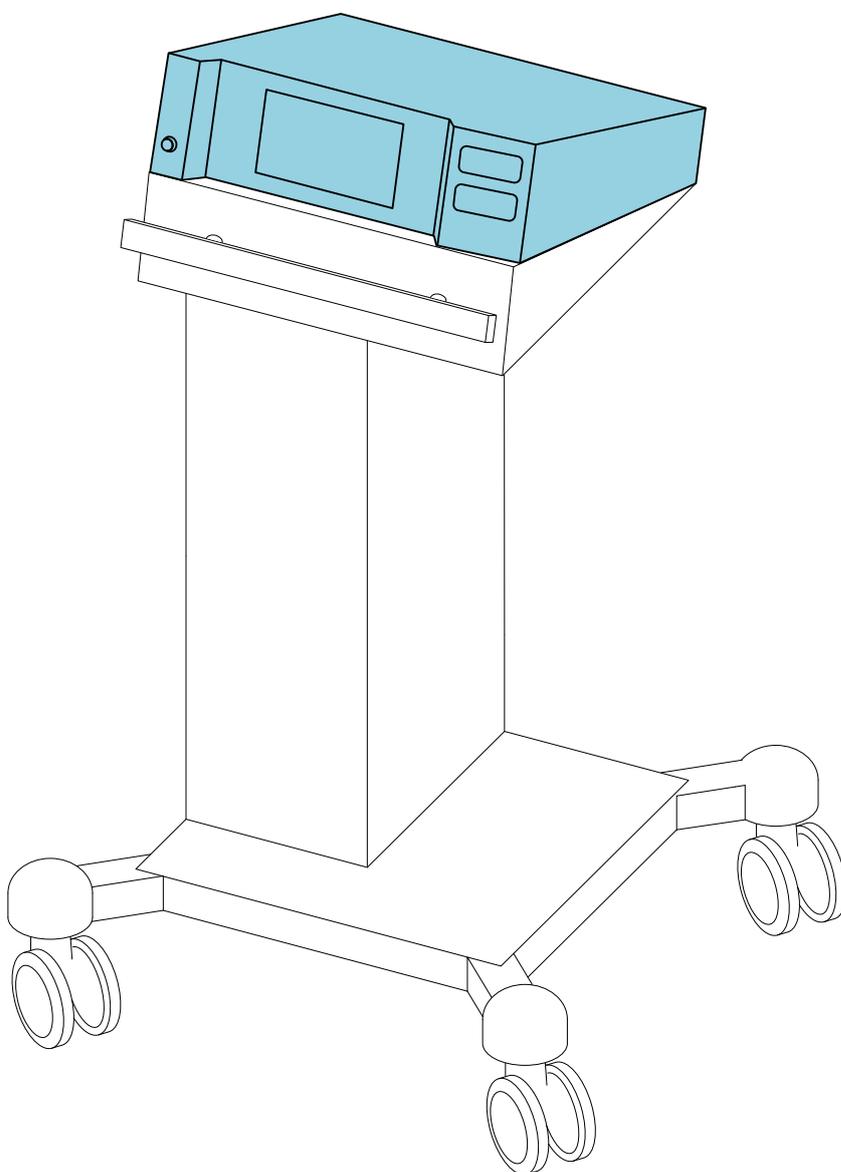---


## Erbe fastening set

**IMPORTANT!** Only use original Erbe accessories.


### Erbe REF 20180-150

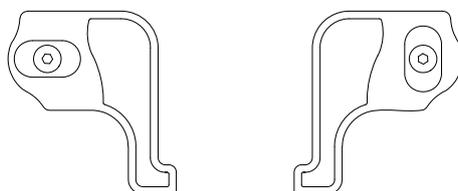M4 x 8

M4 x 16 screws included

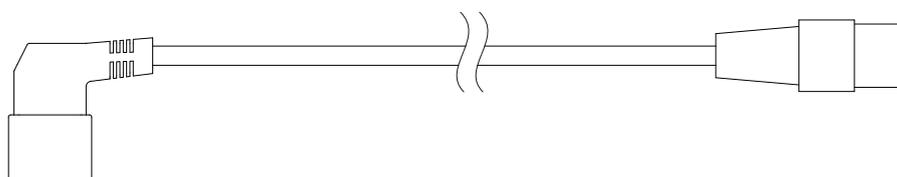Keep for later use







Keep for later use




## Tools required

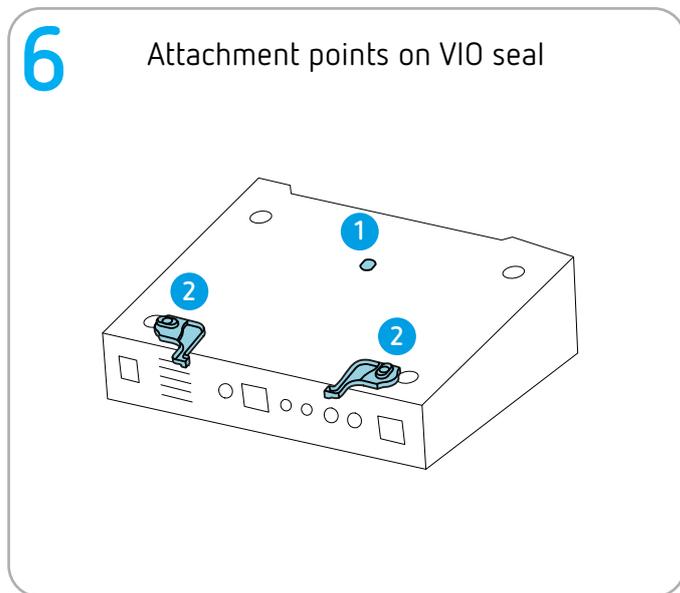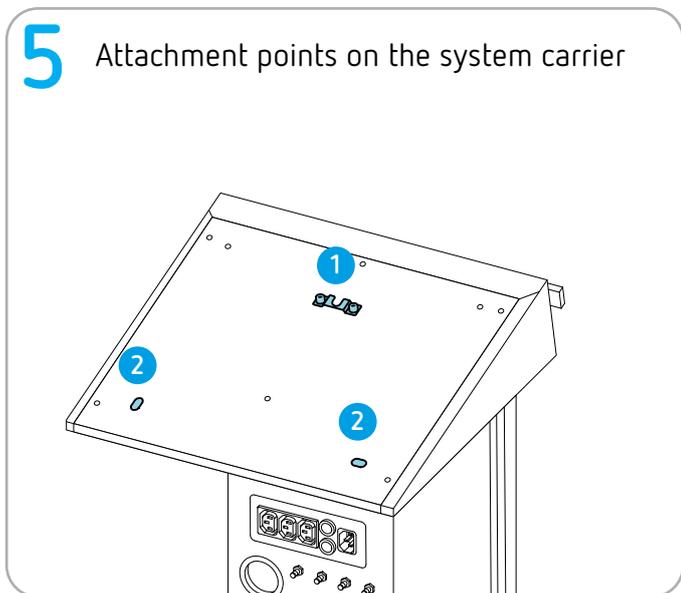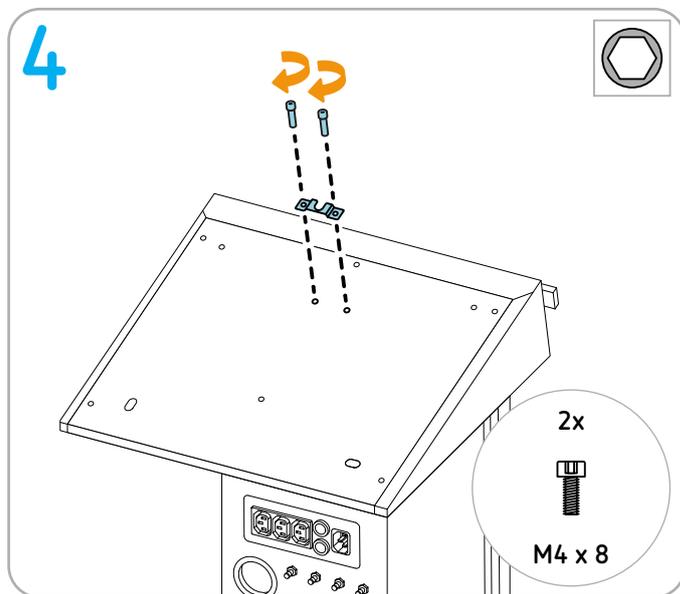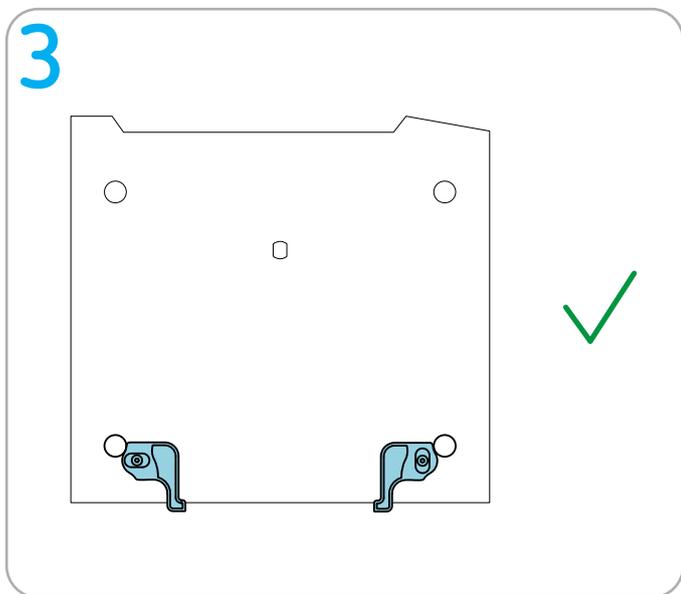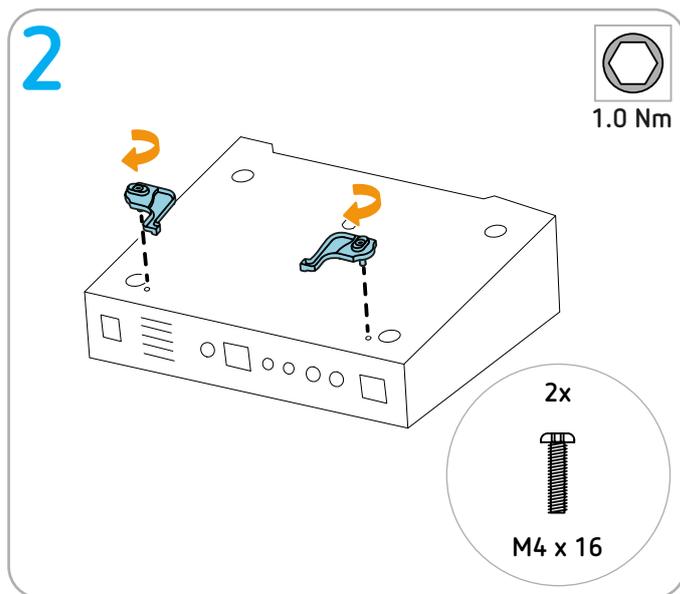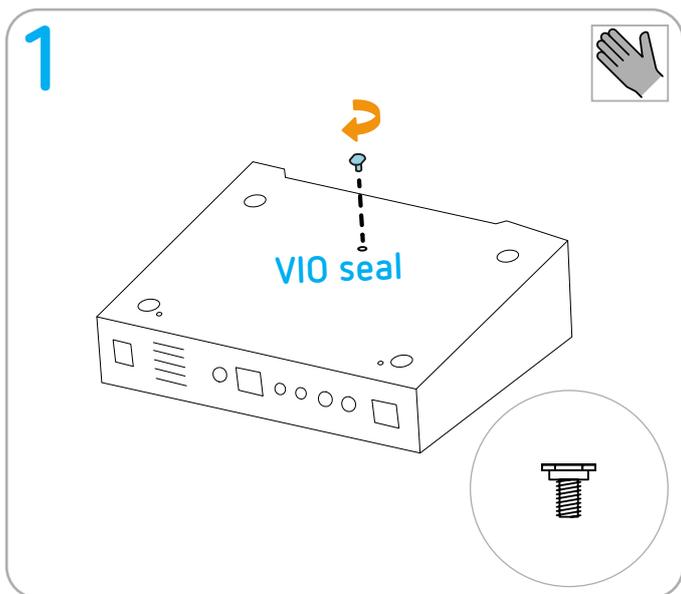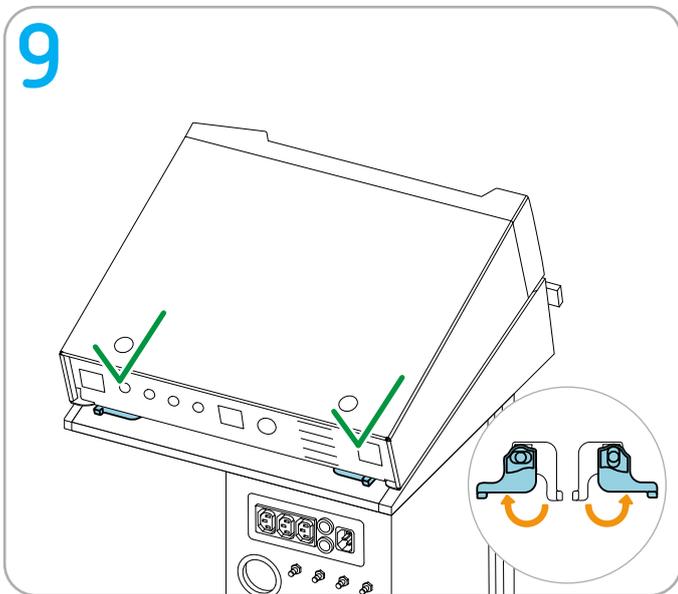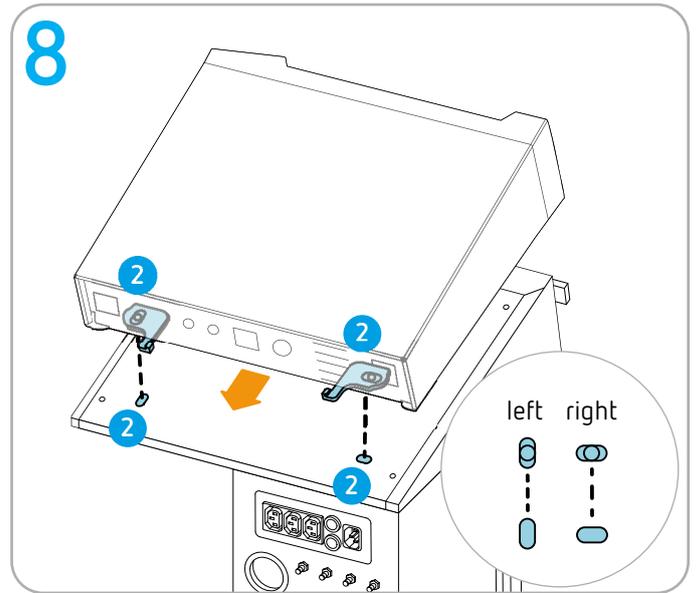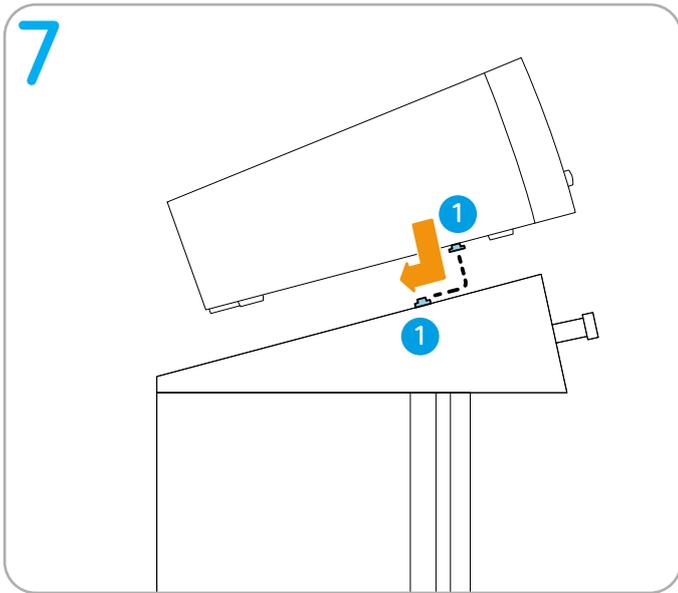-  Torque screwdriver (1 Nm) hex socket 2.5

## Preparation

**IMPORTANT!** Before installing the units on the system carrier, remove all cables, tubing, and instruments.


# Only compact plus: Installing APC 2 and VIO S or VIO D series

---

## Permitted positioning

---

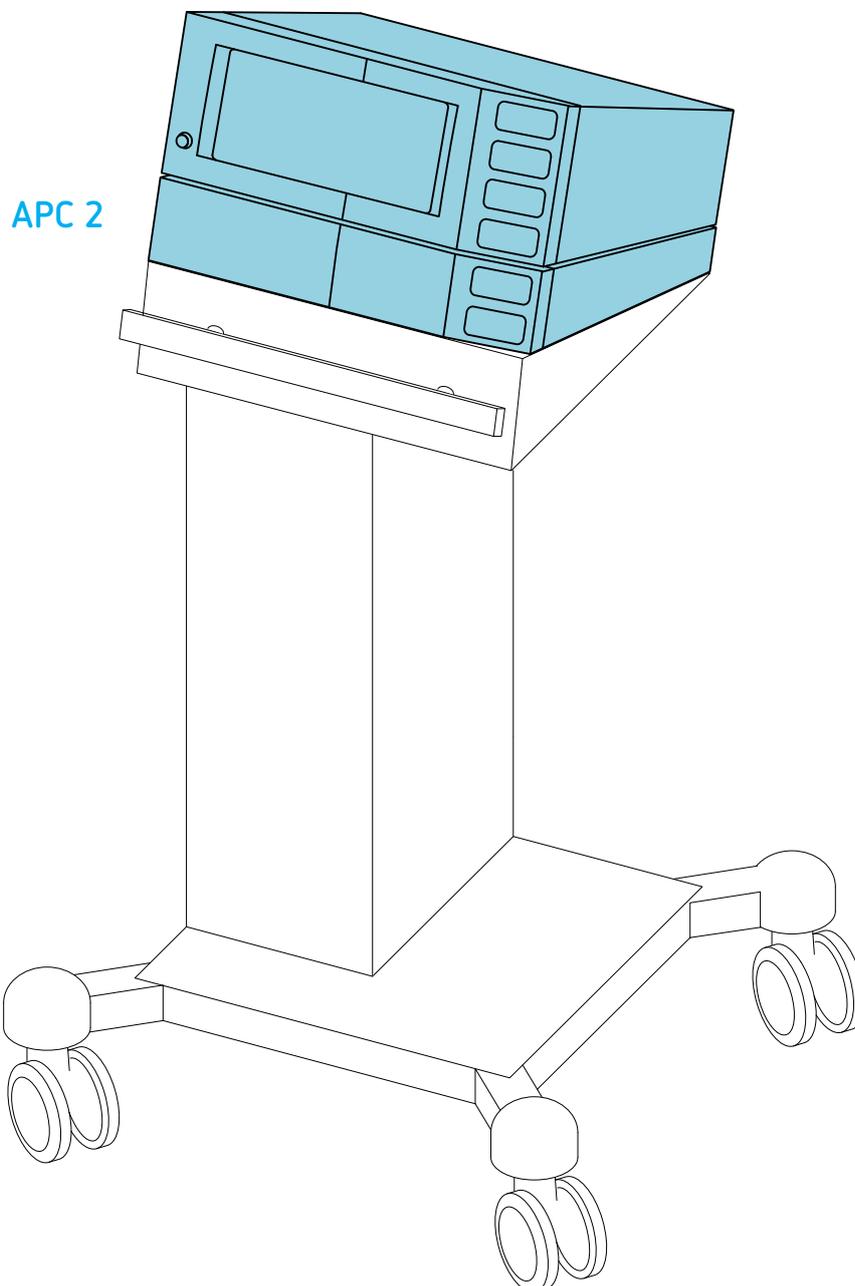VIO S series and VIO D series

---

## Tools required

---

 Hex socket 2.5

---

## Preparation

---

**IMPORTANT!** Before installing the units on the system carrier, remove all cables, tubing, and instruments.

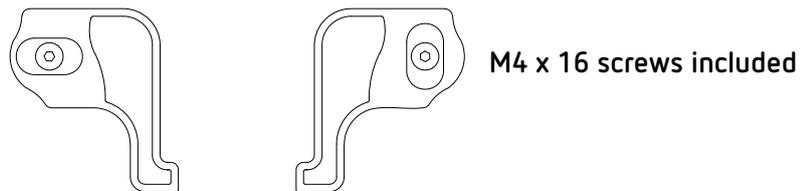
---

## Erbe fastening sets


---







**IMPORTANT!** Only use original Erbe accessories.

**Fastening set for APC on system carrier**  
**Erbe REF 20180-150**

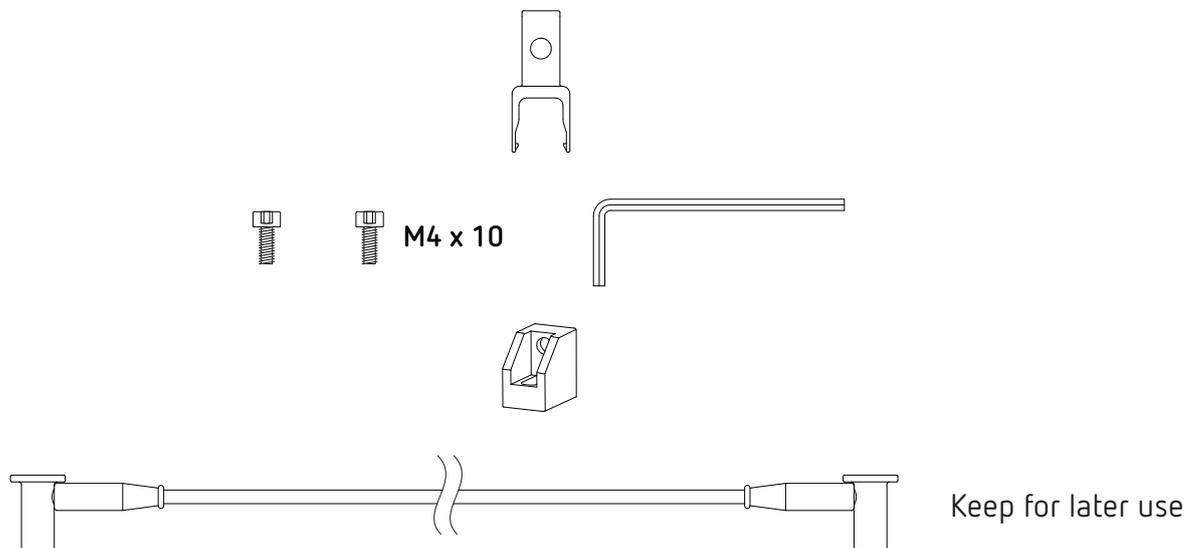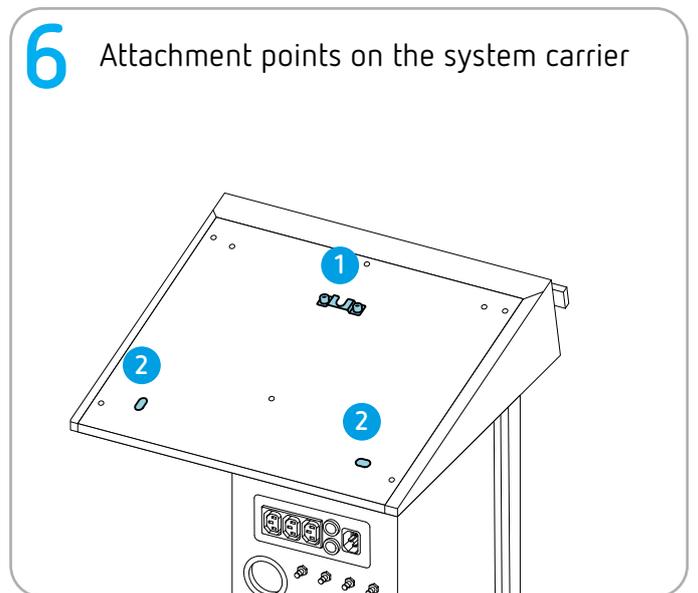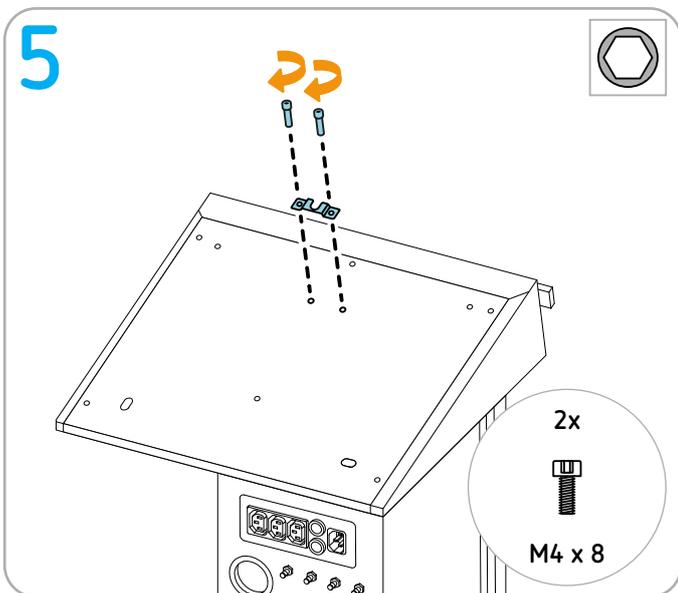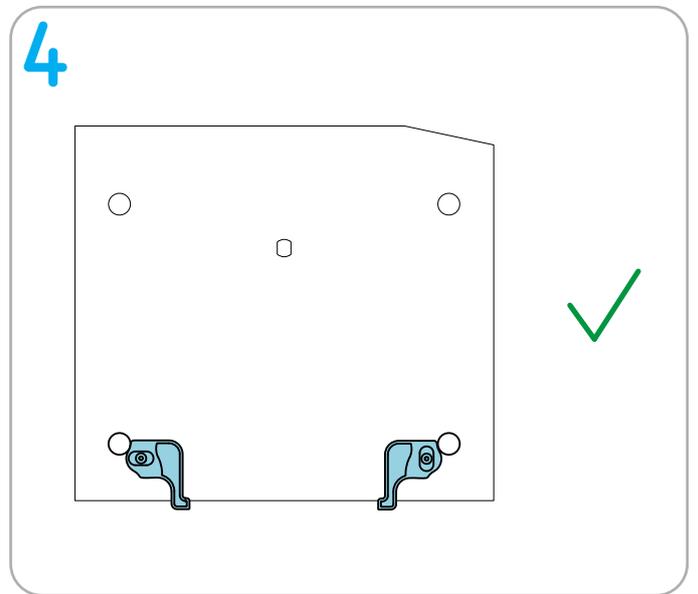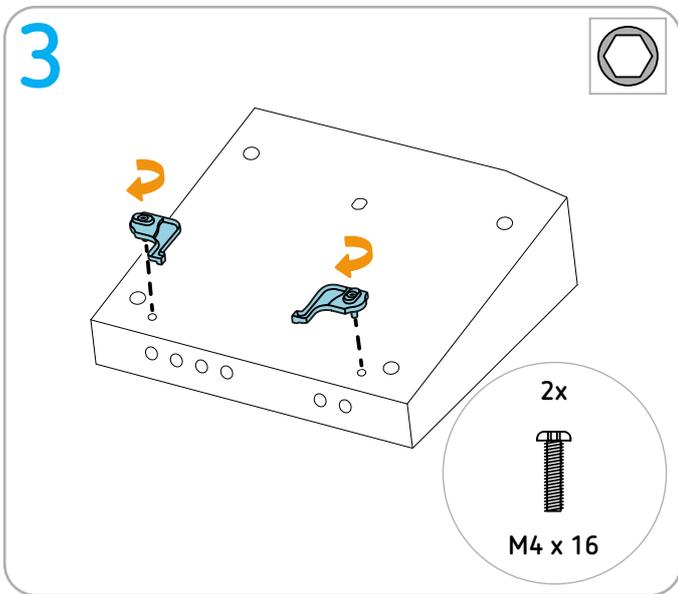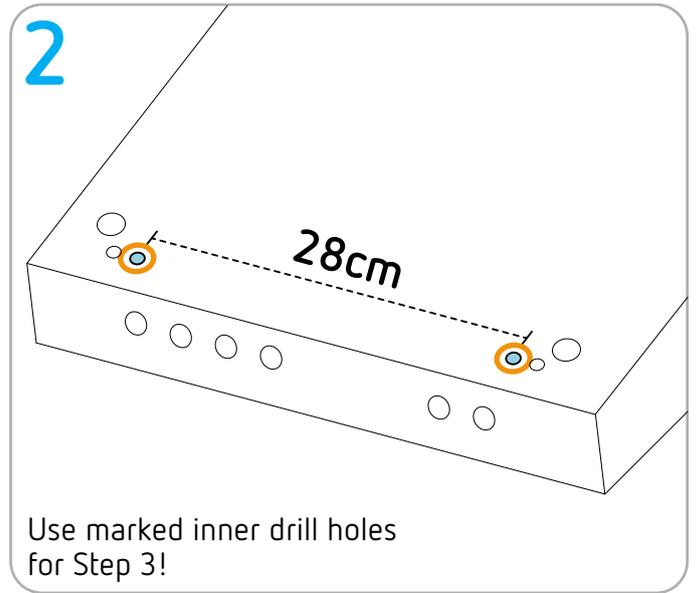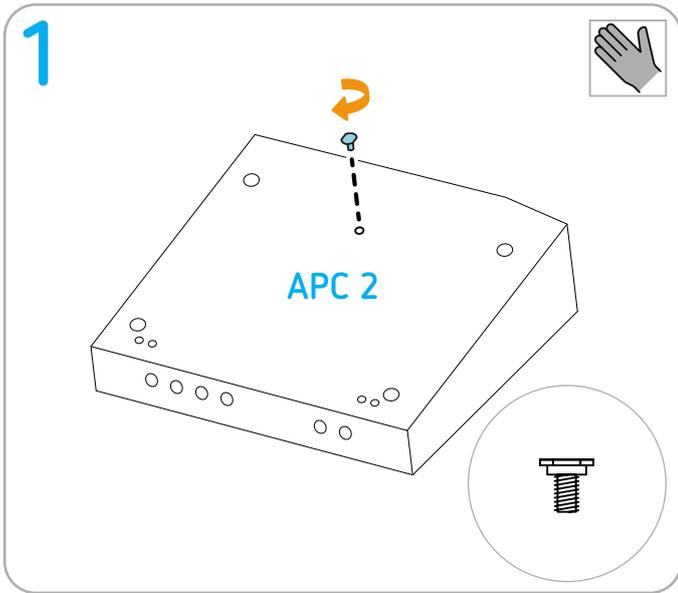
**Fastening set for VIO on APC  
Erbe REF 20180-134**



# 7 Attachment points on the unit

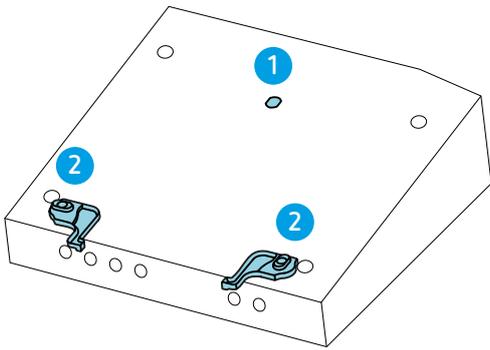
# 8



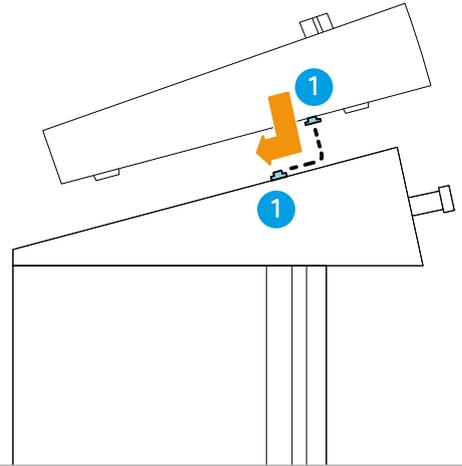
# 9
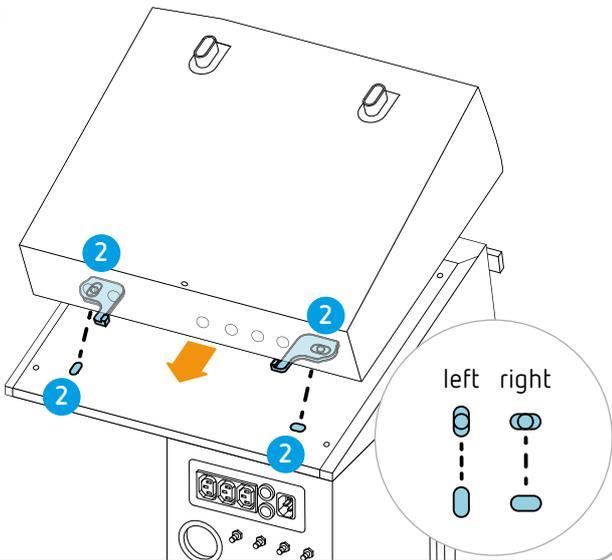
# 10

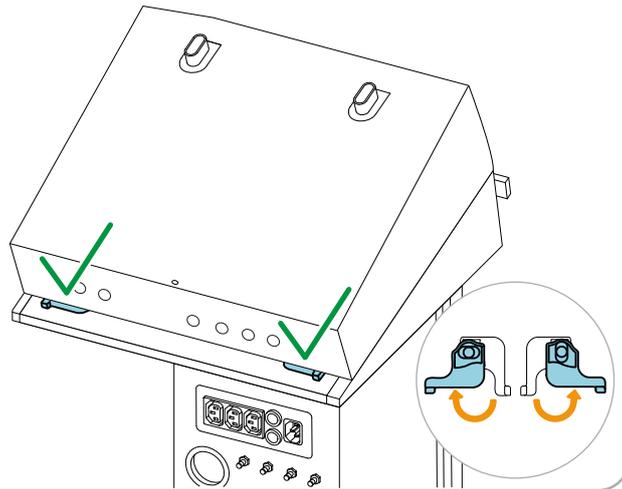Continue to next page

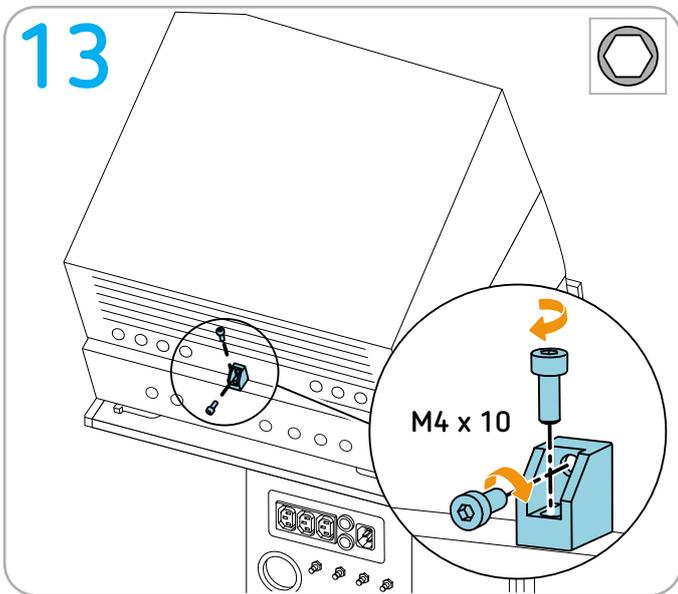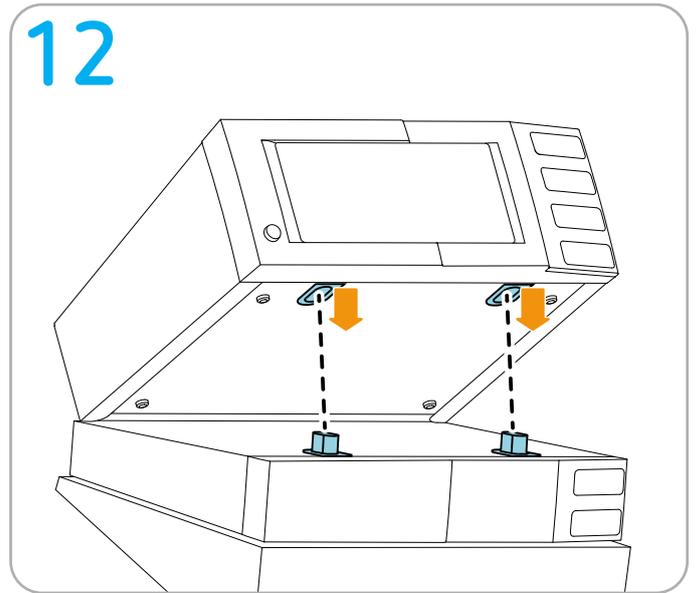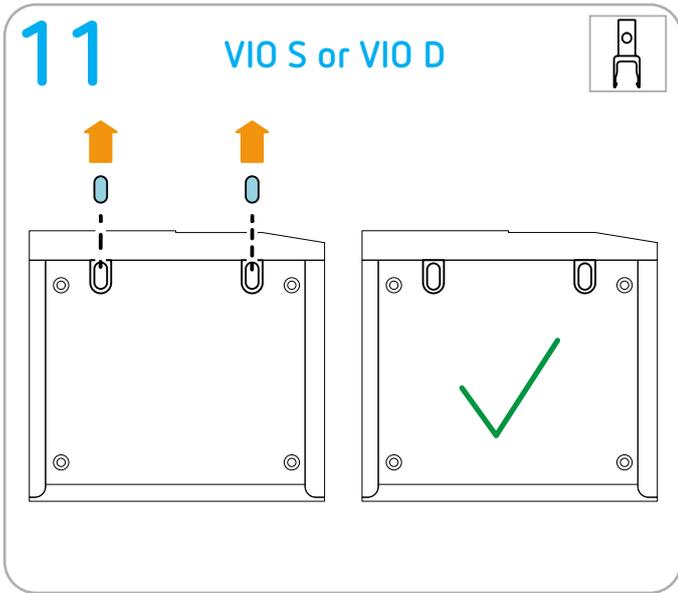
# Only compact plus: Installing APC 3 and VIO 3 series

---

## Permitted positioning

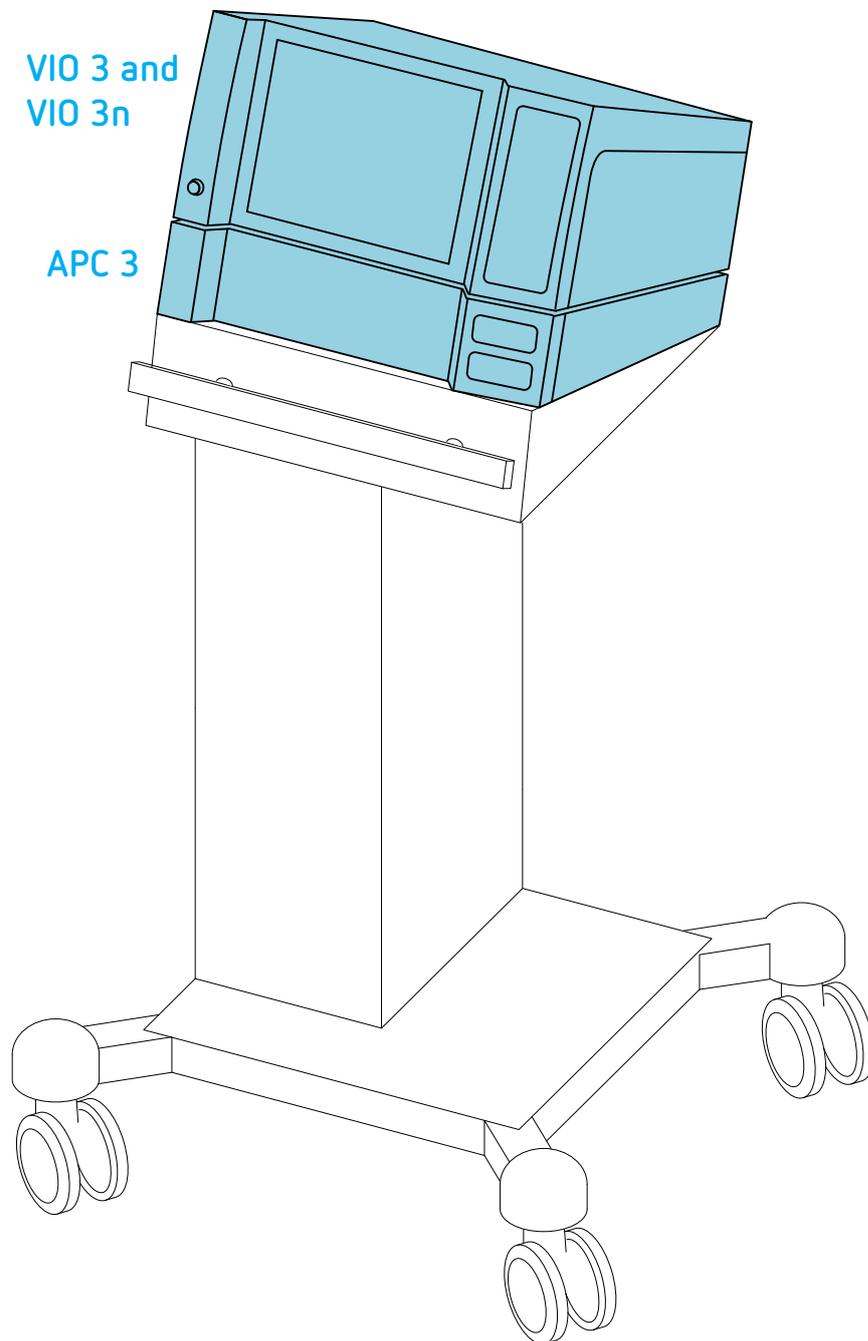---

---

## Tools required

---

 Hex socket 2.5

---

## Preparation

---

**IMPORTANT!** Before installing the units on the system carrier, remove all cables, tubing, and instruments.

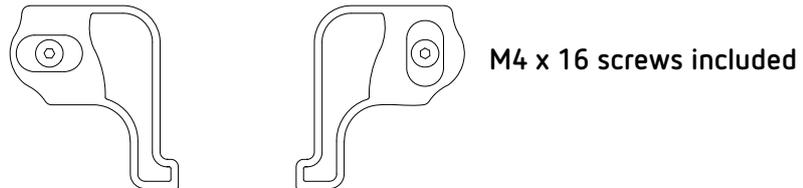
---

## Erbe fastening sets


---

**IMPORTANT!** Only use original Erbe accessories.

**Fastening set for APC on system carrier**  
**Erbe REF 20180-150**

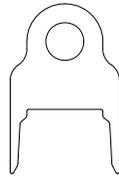**Fastening set for VIO on APC: Erbe REF 30135-030**  
**Bracket: Erbe REF 40160-065**


**M4 x 8**


**M4 x 16**









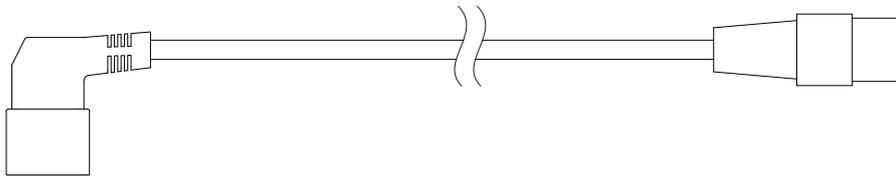
Keep for later use





Keep for later use

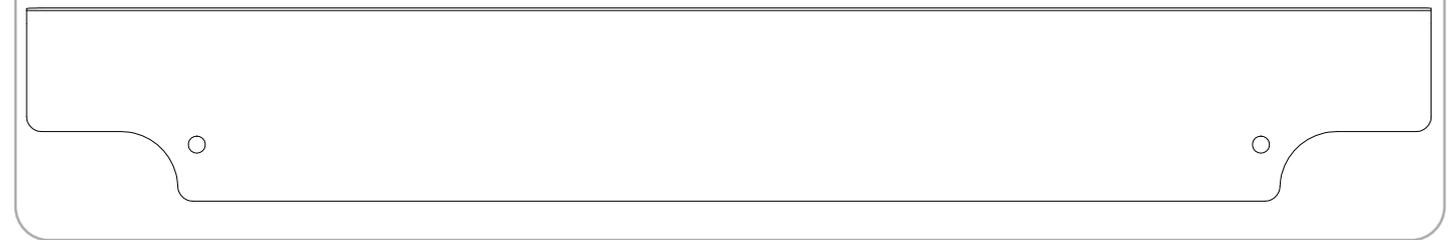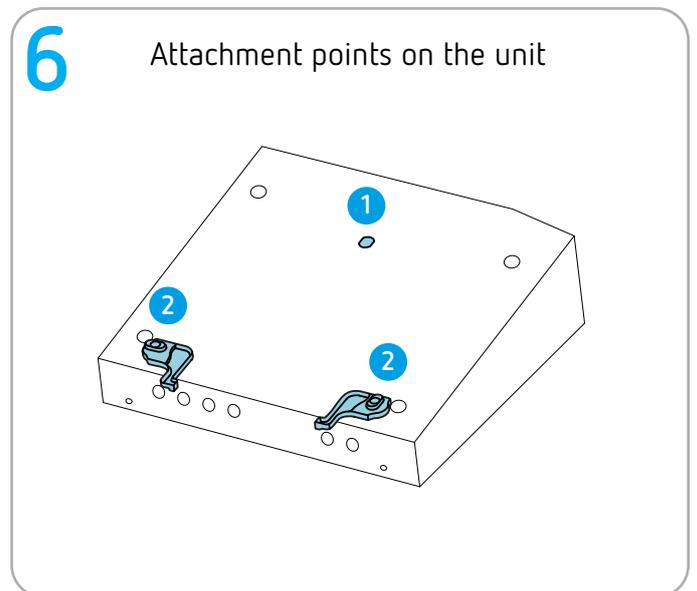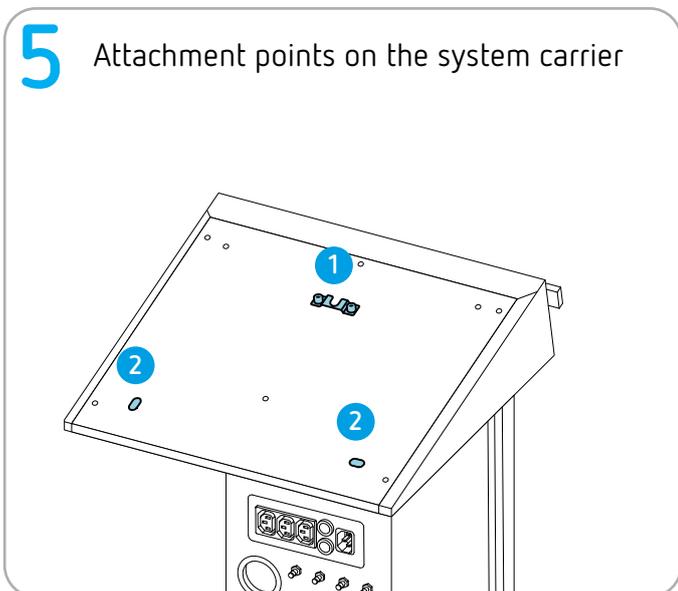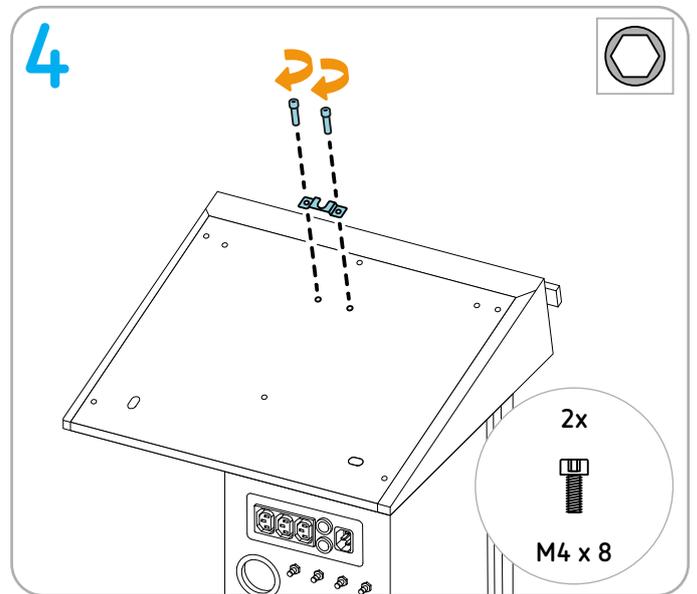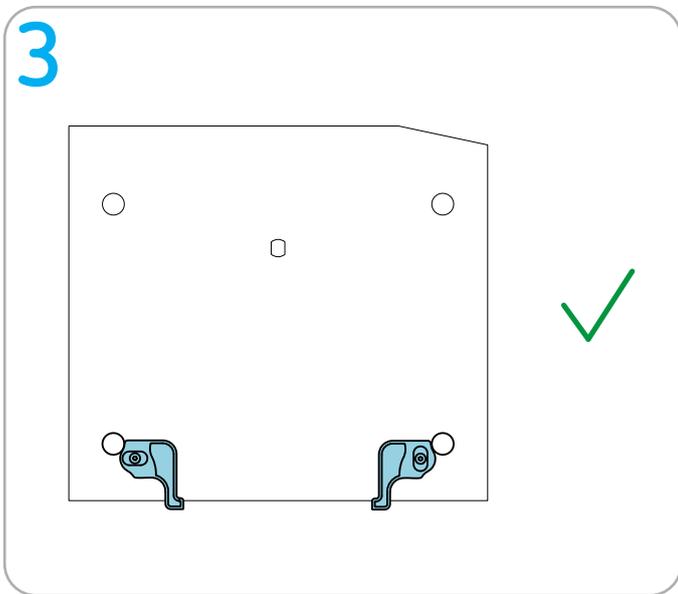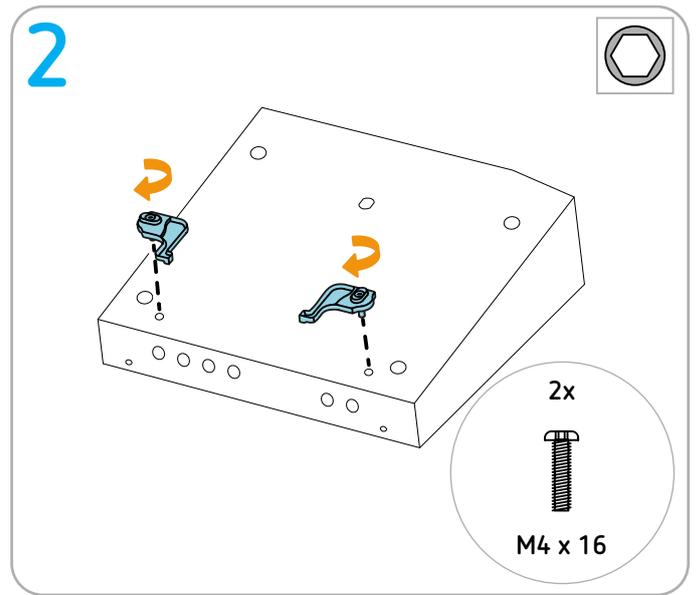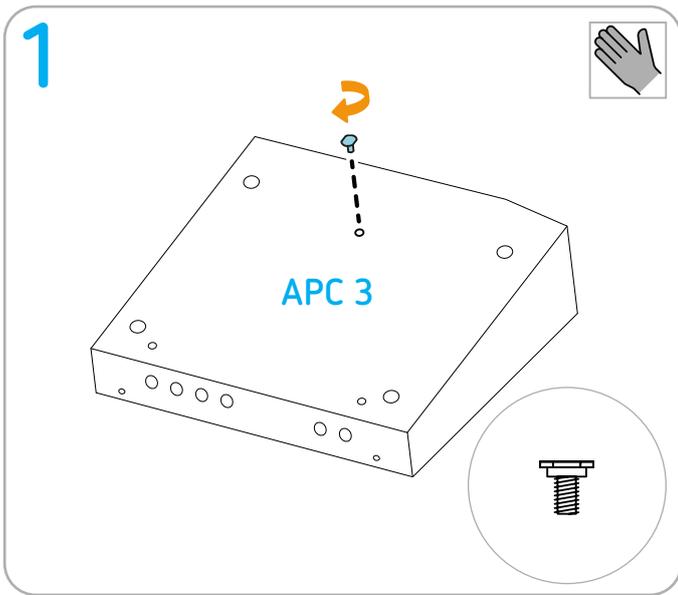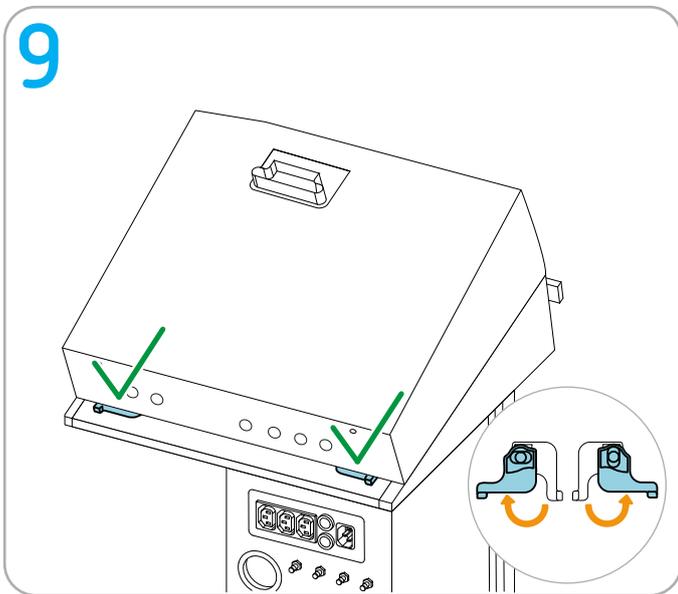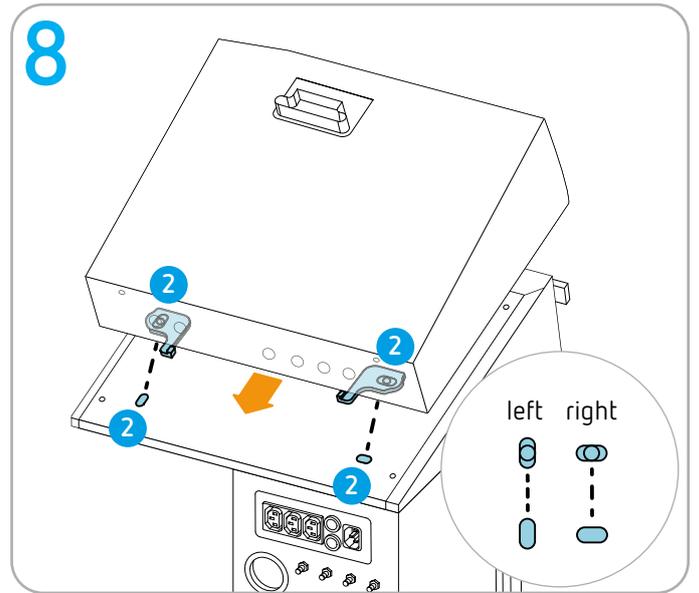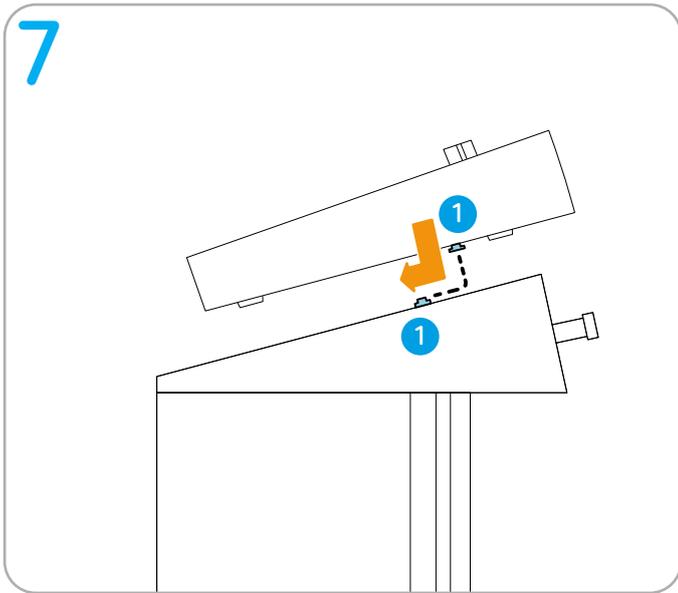Continue to next page

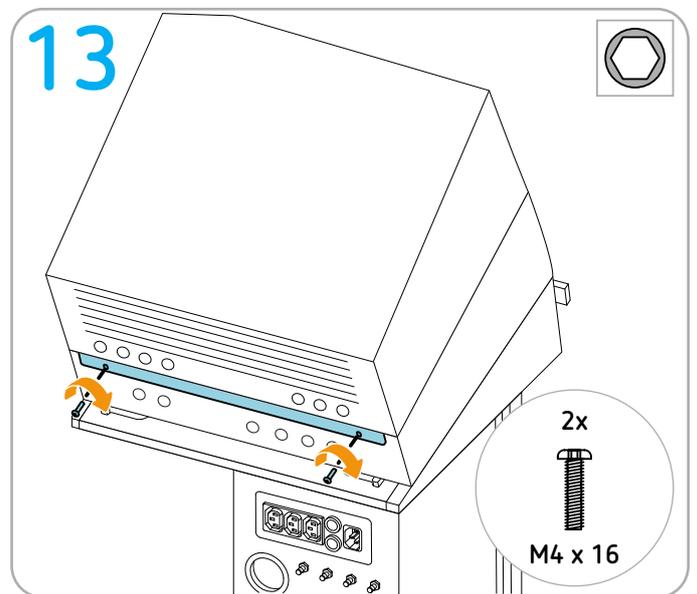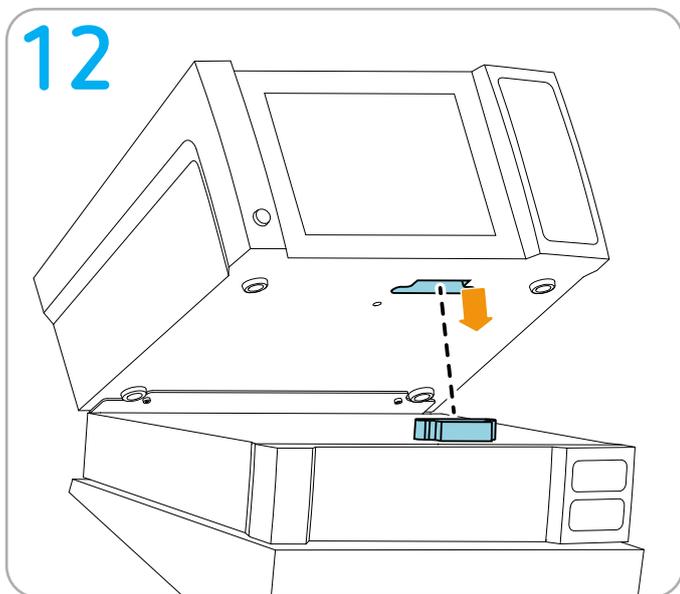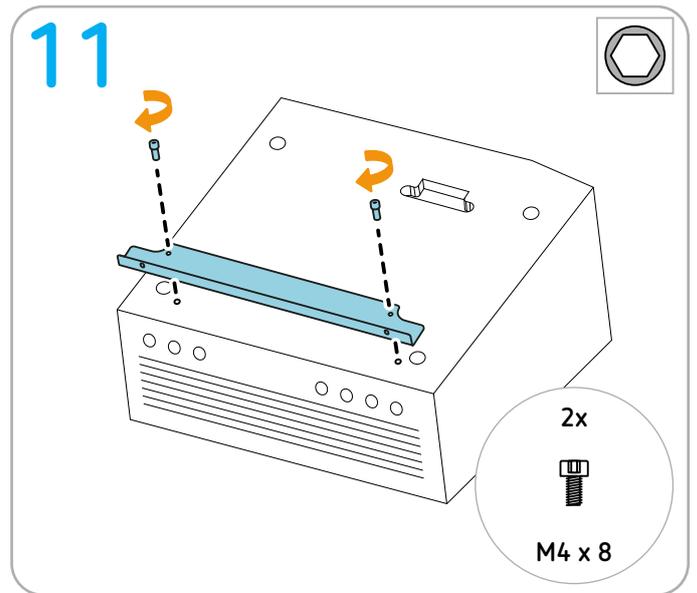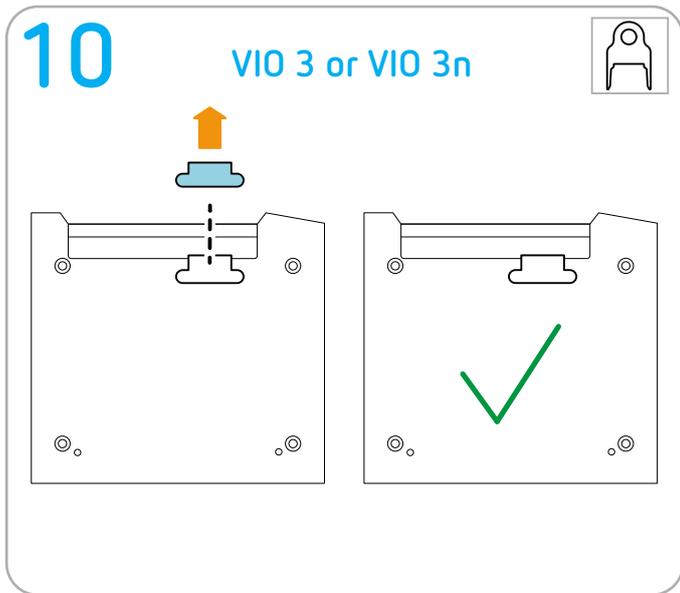
# Only compact plus: Stowing argon gas bottle

---

## Permitted positioning

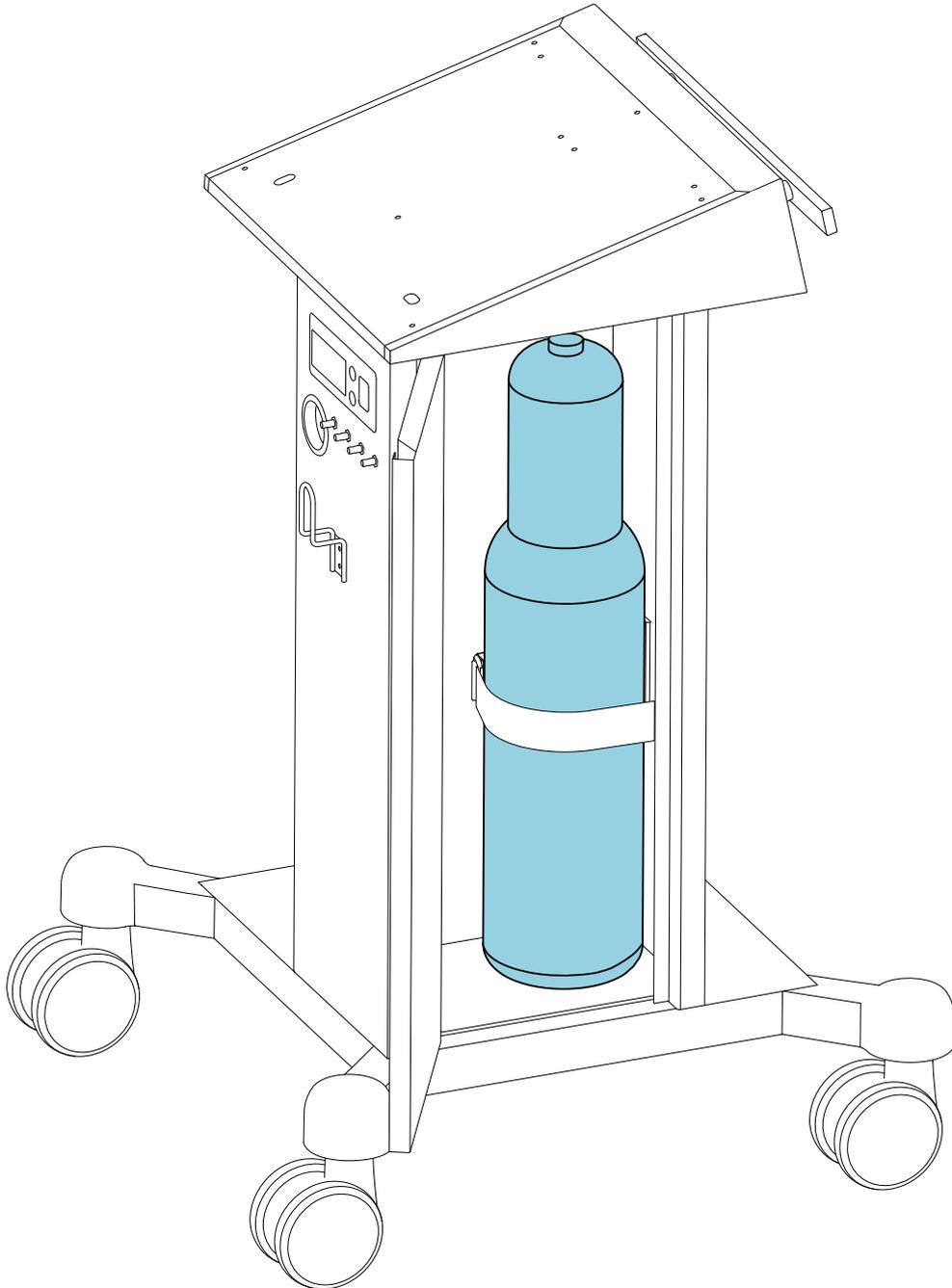---

---

## Tools required

---

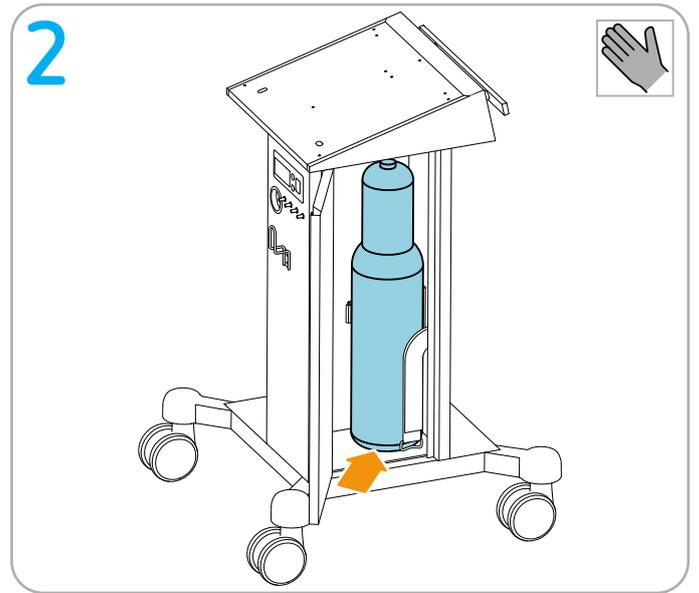
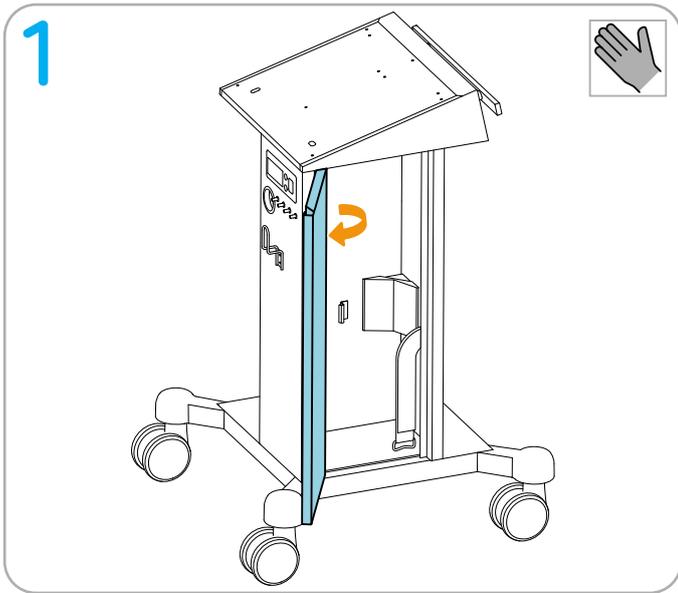
No tool needed.

---

## Preparation

---

**IMPORTANT!** Remove the power cord from the system carrier.


## ECB connection (optional, for communication between the units)

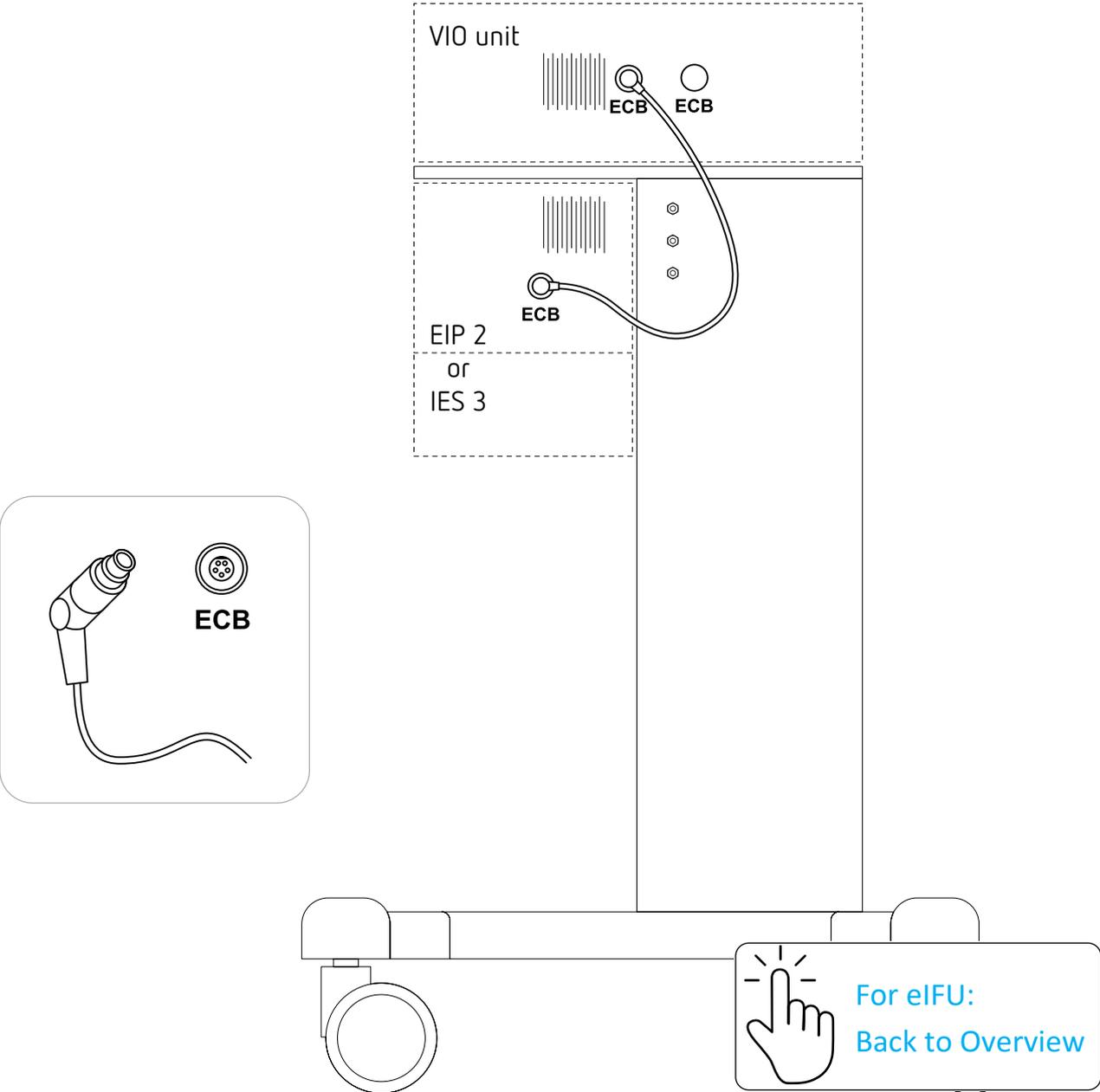
**IMPORTANT!** Only use original Erbe cables.

# compact plus: Making connections


## Potential equalization and mains connection

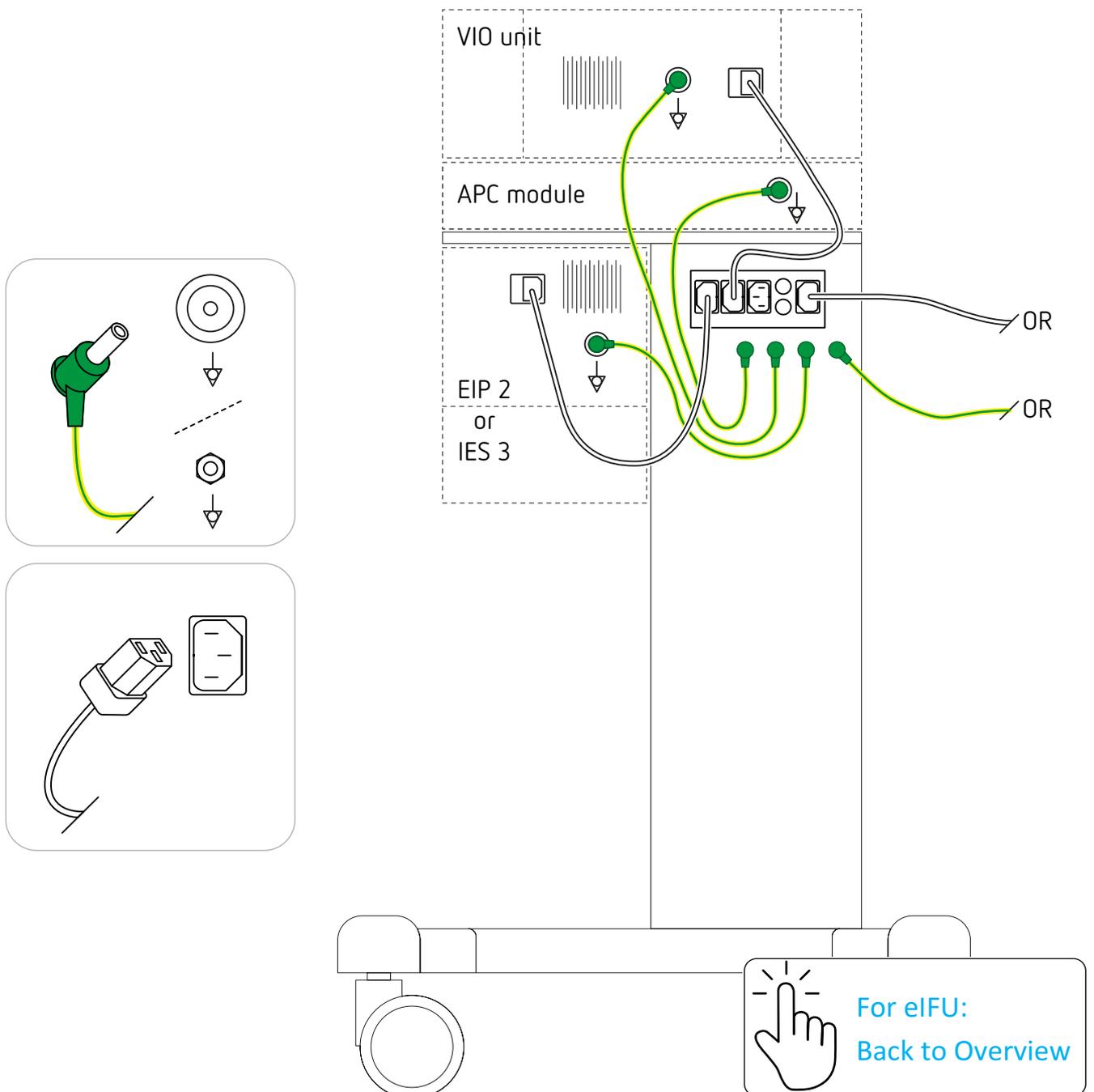**IMPORTANT!** Only use original Erbe cables.

## ECB connection (optional, for communication between the units)

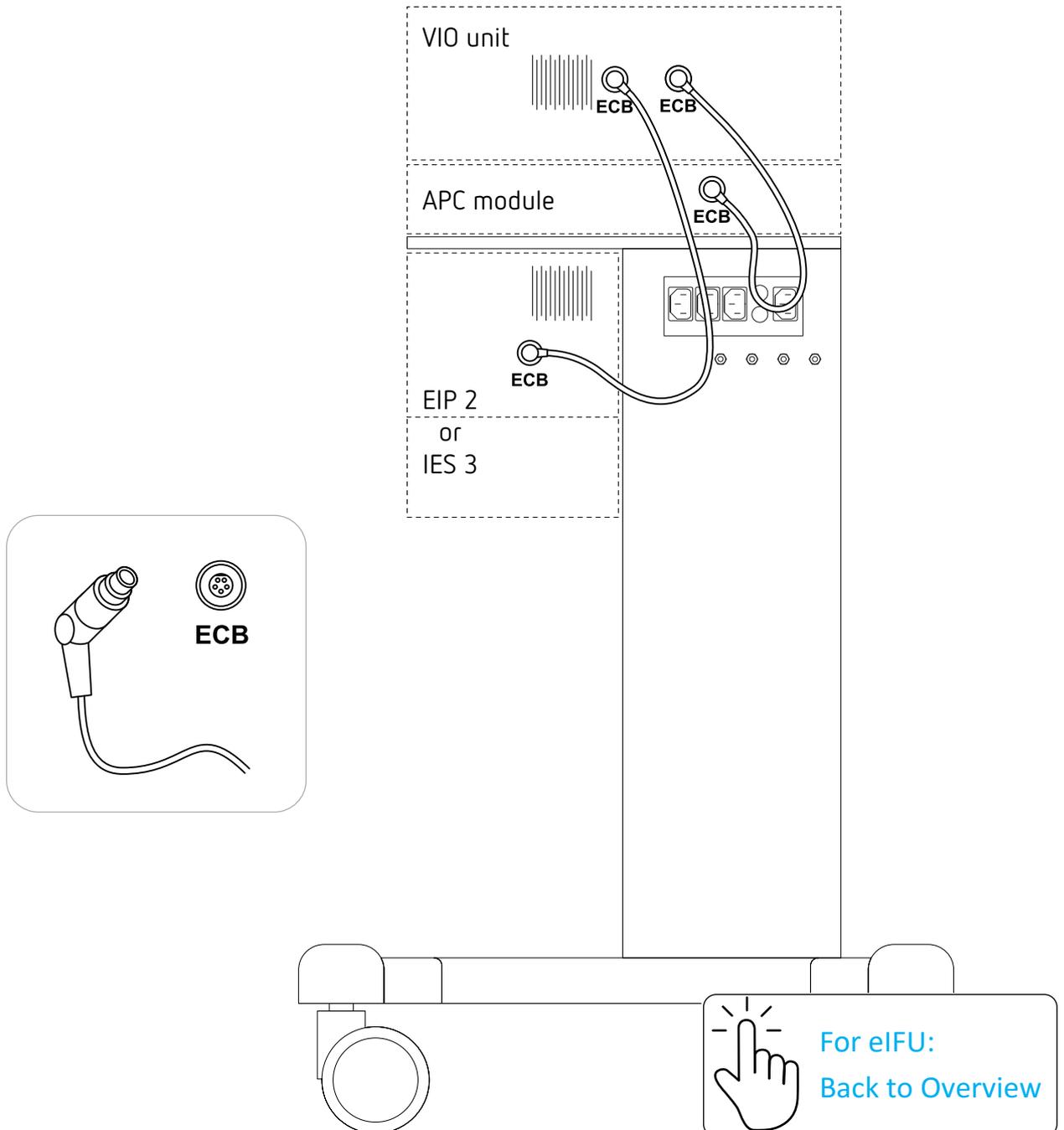**IMPORTANT!** Only use original Erbe cables.

---

## APC connection

---

Please read the User Manual for the APC 2 or the User Manual for the APC 3.

 **Manufacturer**  
Erbe Elektromedizin GmbH  
Waldhörnlestr. 17  
72072 Tübingen, Germany  
erbe-med.com

**Service Hotline**  
Tel +49 7071 755 437  
techservice@erbe-med.com

**Service Hotline USA**  
Tel +1 770 955 4400  
or +1 800 778 3723  
technicalservice@erbe-usa.com